seca **727/728**

Deutsch	Seite 3
English	Page 5
Français	Page 7
Español	Página 9
Italiano	Pagina 11
Dansk	Side 13
Svenska	Sida 15
Norsk	Side 17
Suomi	Sivu 19
Nederlands	Pagina 21
Português	Página 23
Ελληνικά	Σελίδα 25

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Transportsicherung

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
Preparation is important...

Transport locking device

Loosen transport locking device. First turn over the scales so that the baseplate is at the top. Loosen the two outer clamping screws (1). Pull out the spacers (3) between the frame (5) and the cradle support (2) sideways and remove the clamping bar (6) with the screws and spacers. Note: Before transporting over long distances, the scales must be locked again by re-inserting the spacers and tightening the clamping screws hand tight. Therefore keep the transport locking device and packaging.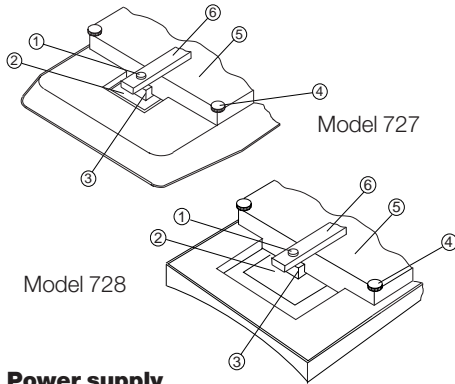

Power supply


The scales can be either mains or accumulator operated. A power supply unit and a set of accumulators are supplied; both parts are connected by the following procedure. You can then choose between mains and accumulator operation.

Mains and accumulator operation

Insert the low voltage plug of the plug-in power supply unit provided into the low voltage socket (9) of the baseplate.

Press lock (8) in the cover of the battery compartment towards the text "BATTERY" and open the cover. Then insert the plug (7) of the set of accumulators (10) into socket (11). Insert the set of accumulators in the battery compartment, close the cover again and engage the lock.



Note: For weighing independently of the mains, the set of accumulators must be charged up before initial use by plugging the power supply unit into the scales for 24 hours. Connect the plug-in power supply unit to the mains.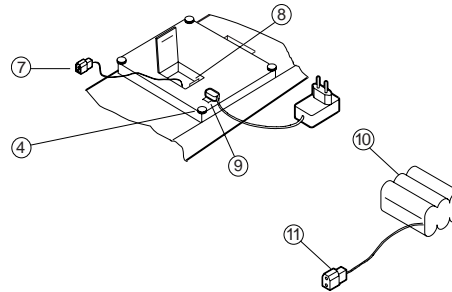

How to weigh correctly...

Press the green start button (13), **START** and then **0.000** appears in sequence on the display (16). The scales are ready for operation. Zeroing takes place automatically when the scales are empty. However, if a value other than **0.000** appears on the display, the start button (13) is to be pressed for zeroing prior to weighing. The empty scales switch off after 10 minutes when mains operated, and after one minute when accumulator operated. Before using the scales for further weighing, the start button must be pressed again and **0.000** awaited.

With accumulator operation, **1.74: 1** appears on the display if the accumulators have to be recharged. This happens automatically by connecting the plug-in power supply unit provided.

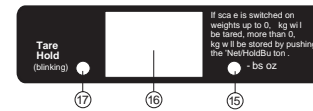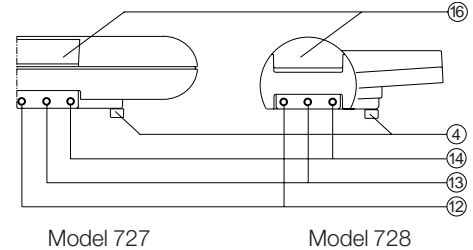Changeover kg/lbs

If you require the weight to be displayed in lbs, press the kg/lbs changeover switch (14) after switching on or while weighing. The lbs indicator (15) lights up for the lbs setting.

On scales with the Autohold function

The Autohold function is intended to save the user pressing the Hold key.

It is active as soon as the scale is switched on. If a load of over 400 g is placed on the scale, the weight is automatically held in the display. Pressing the Hold key releases the weight from Hold.

The Autohold function only becomes active again when a load of over 400 g is placed on the scale. This procedure can be repeated as often as required.

Taring device

When the scales are switched on, there is provision for taring items such as cushions and towels, which have been placed on the scales before weighing, up to a weight of 400 grammes (14 oz).

1. Place cushion or towel on the scales – the weight is displayed. Press Tare/Hold button (12) –  appears and the indicator for Tare/Hold (17) lights up. The actual weighing process can now take place.
2. When the scales are completely empty,  appears automatically and the indicator for Tare/Hold (17) goes out.


Prolonging the weight display

The scales offer the facility for retaining weights in excess of 400 grammes on the display after the weighing process.

1. Switch on scales with the start button (13).
2. Place the baby on the scales – the weight is displayed.
3. Press the Tare/Hold button (12) and the weight is stored. The indicator for Tare/Hold (17) flashes.
4. If the baby is taken off the scales, the weight continues to be displayed.
5. The permanent display is cancelled by pressing the Tare/Hold button (12) or the start button (13) again. The indicator for Tare/Hold goes out.

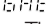
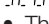
2g/5g resolution changeover switch

To change over the resolution from 5g to 2g, please open the battery compartment. The switch (A) is located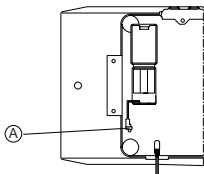

at the end of the cable in the battery compartment. The 2g resolution works up to a weight of 10 kg. Then 5g resolution takes over.

What do I do if...

- ... The display remains dark?
 - Check the mains connection
 - Check the connection of the low voltage plug (9).
- ... No weight is displayed under load?
 - Check whether the transport locking device has been removed
- ...  appears on the display?
 - Press start button.
 - If taring, press the tare button.
- ...  appears on the display?
 - The accumulators must be recharged.
- ...  appears on the display?
 - The scales are overloaded, reduce load.
- ...  appears on the display?
 - Check whether scales are overloaded.
 - The automatic self-checking system has found a fault. Inform seca service department.

Technical data

Weighing range: 0 - 20 kg 0 - 44 lbs

Fine graduation: 5 g 0.2 oz

In 2g graduation mode

0 kg to 10 kg: 2g 0.1 oz

10 kg to 20 kg: 5g 0.2 oz

Tare range: 0 - 4 kg 0 - 14 oz

Permanent display: 0.4 - 20 kg 14 oz - 44 lbs

Height of mod. 727 151 mm 6"

Height of mod. 728	163 mm	6 1/2"
Width	551 mm	21 5/8"
Depth	320 mm	12 5/8"
Empty weight:	5 kg	11 lbs

Power supply

Power supply unit depending on model

230 V~/12 V~/130 mA
(seca no. 68-32-10-252)

100-240V~/50-60Hz/12V~/0.5A
(seca no. 68-32-10-265)

Accumulator: 6 unit accumulator set
(seca no. 68-22-12-721)

IEC 601-1: class of protection II, type B

We only provide a guarantee for the safety, reliability and performance of the equipment if:

- the equipment is used in conformance with the operating instructions, and
- the power supply unit provided (12V/130 mA, seca no. 68-32-10-252) or the 6 unit accumulator set supplied is used, and
- repairs are undertaken by authorised persons and
- the transport locking device and packaging supplied are used for transport and
- the scales are only cleaned with normal disinfectants.

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Gravitational compensation

Instructions for gravitational compensation (gravitational compensation value)

The force of gravity is not the same everywhere, but depends on the geographic latitude and the height above sea level. The seca 727 baby scales work so accurately that these differences are visible through differences in the measured value on the display. These differences can be easily compensated by aligning the arrow of the adjusting screw shown in figure 1 with the scale value determined from the table (gravitational compensation value).

Example: Installation point = Sydney, (1) degree of latitude = 35°, (2) height above sea level = 100 m, scale value determined according to table = 4, the scales are adjusted to the value 2 at the factory. No adjustment is required within the entire zone 2.

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**seca 727 baby scale
with RS232 serial
interface**

Underneath the scale is a serial interface which allows the scale to be connected to a PC. The interface cable required is available as an accessory. **Detailed information can be obtained from seca Customer Service.**

**Spare parts and
accessories**

Accessory:
model 451 interface cable
Spare part no.:
08-06-16-085.

Detailed information can be obtained from seca Customer Service.

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Warranty

A two-year warranty from date of delivery applies to defects attributable to poor materials or workmanship. All moveable parts – batteries, cables, mains units, rechargeable batteries etc. – are excluded. Defects which come under warranty will be made good for the customer at no charge on production of the receipt. No further claims can be entertained. The costs of transport in both directions will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be honoured only if the complete original packaging was used for any transport and the scale secured and attached in that packaging just as it was when originally packed. All the packaging should therefore be retained.

A claim under warranty will not be honoured if the equipment is opened by persons not expressly authorised by seca to do so. We would ask our customers abroad to contact their local sales agent in the event of a warranty matter.

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GB

We hereby certify that the above designated product meets the basic safety requirements laid down in the Directive of the Council pertaining to the harmonisation of the legal provisions of the member states in respect of electro-magnetic compatibility (2004/108/EC).

⚠ If temporary interference and/or inaccurate weighing should nevertheless occur when the scale is used in the vicinity of intensive electromagnetic radiation sources such as microwave appliances, mobile telephones, walkie-talkies and the like, please move the scale away from the interference source or switch off the interference source and repeat the weighing operation.

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