


SensaTrak

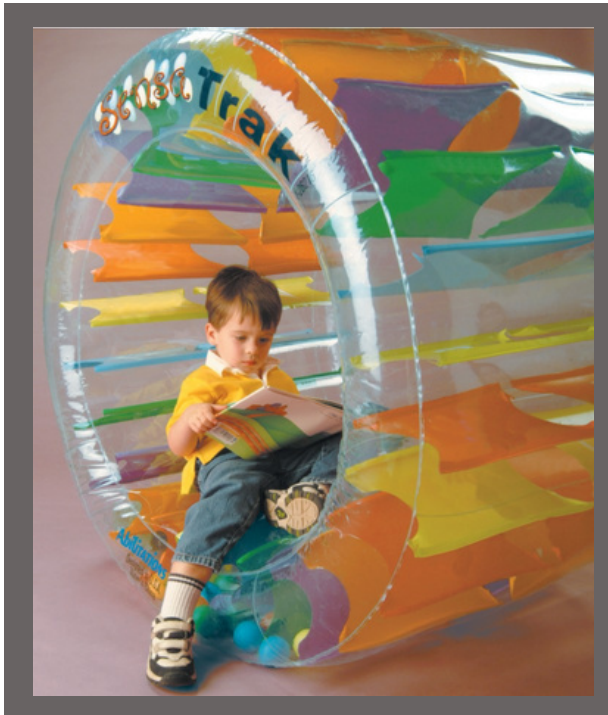

Item 1376273

SensaRock



Item 1376274

Introduction

The SensaTrak and SensaRock are among the most visually, vestibularly, tactilely, and auditorily stimulating items available. Each is filled with colorful balls that provide visual, tactile and auditory effects as they flow through the colored chambers. Kids, therapists and parents have all fallen in love with these products, and you will too! Just the slightest movement inside the air-filled tube provides children with a variety of motor and sensory input while challenging their reactions.

Product Setup

These products are intended to be used with a power inflator (not included). Use the narrow tip attachment, and push the tip into the valve, past the "flap". Inflate to the desired firmness, but **DO NOT OVERINFLATE**. Overinflating could cause the product to burst.

Care and Safety

- **WARNING:** This product is intended to be used on dry ground **ONLY**. It is **NOT** intended as a water float, and has not been safety tested for such use.
- Clean with a mild soap solution and water.
- Sanitize with mild bleach solution.