

1. Remove top wheelchair seat back screws, place brackets as shown in diagram **A**. Reinstall screws and tighten while holding brackets in proper position.
2. Suspend nylon cylinder tank holder top straps as shown in diagram **B**. Install bottom straps to wheelchair frame allowing enough slack to keep tank away from back of chair.
3. When using in conjunction with Safe•t mate Anti-rollback Models SM-001, SM2, SM2-2, adjust position left or right to keep cylinder tank holder away from moving parts.

CAUTION:

*Wheelchair rear anti-tippers are recommended.
(i.e. Safe•t mate Fully Adjustable Wheelchair Rear Anti-tippers, Model SM-008)*

