

ABSORBENT WOUND DRESSING

SOFSORB®

SOFSORB®



- Non-adherent
- Highly absorbent multi-layer dressing
- Can be used wet or dry for the treatment of various wounds

 DeRoyal®



ABSORBENT WOUND DRESSING

SOFSORB

FEATURES & BENEFITS

- A highly absorbent, multi-layered one-piece dressing
- Five layer construction:
 - Non-adherent wound contact layer
 - Stay-Dry Liner that permits passage of wound drainage to keep peri-wound skin dry and helps to prevent maceration at wound site
 - Center layer that absorbs wound drainage
 - Cellulose layer that wicks drainage horizontally along pad to increase dressing absorption capacity
 - Air-permeable backing for strength and durability
- May be soaked with saline or topical agents prior to use
- Rounded edges and tapered limb configurations conform to curved and hard-to-dress areas without unnecessary bulk
- Ideal for protecting wound sites from further trauma
- Soft pliability allows for range of motion exercises during recovery

5 LAYER CONSTRUCTION



1. Non-adherent wound contact layer
2. Nonwoven layer permits passage of wound drainage to keep peri-wound skin dry
3. Center layer absorbs wound drainage
4. Cellulose layer wicks drainage horizontally along the pad to increase dressing capacity
5. Air permeable backing provides strength and integrity

APPLICATION

1. Sofsorb dressing can be used either dry or wet. To use wet, soak with saline or preferred topical solution, then squeeze out excess fluid. Apply the dressing with the wound contact layer toward the wound surface.
2. Secure in place with Stretch Net™ tubular elastic net or with Fluftex™ roll gauze.
3. To remove dressing, carefully lift Sofsorb off the wound. Moisten with saline if necessary. Discard dressing.





ABSORBENT WOUND DRESSING

SOFSORB

INDICATIONS

- Burns
- Post-operative incision site
- Heavily draining skin ulcers
- Heavily draining pressure sores
- Lacerations and abrasions

SIZING

Choose the Sofsorb dressing that best fits the size of the wound from the sizes available. If using a tapered limb Sofsorb dressing, determine the correct size by measuring the largest circumference of the limb with the special measuring tape.



Sofsorb is the ideal dressing for draining ulcers and pressure sores.



Sofsorb can be soaked with saline or preferred topical agents prior to use.



Sofsorb's rounded edges conform well to curved areas.



Sofsorb tapered limb dressing contours to fit wound sites without unnecessary bulk.



ABSORBENT WOUND DRESSING

SOFSORB

STANDARD DRESSING



TAPERED LIMB DRESSING



CUSTOM DRESSING



PADS & SHEETS



Sofsorb Standard Dressings

Product #	Size	Qty/Bx	Qty/Cs
46-101	4" x 6"	30	120
46-102	6" x 9"	20	80
46-103	9" x 15"	n/a	30
46-104	15" x 18"	n/a	20
46-105	15" x 24"	n/a	15
46-106	4" x 9"	25	100
46-109	3" x 3"	30	60

Sofsorb Tapered Limb Dressings

Product #	Size	Qty/Bx	Qty/Cs
46-111	XS (15")	n/a	20
46-112	S (18")	n/a	20
46-113	M (21")	n/a	20
46-114	ML (23")	n/a	15
46-115	L (26")	n/a	15
46-116	XL (29")	n/a	15

Sofsorb Custom Dressings

Product #	Description	Qty/Cs
46-132	Drainage/Tube Dressing	100
46-121	Full Arm Dressing	20
46-122	Full Leg Dressing	20
46-124	Pediatric Boot/Foot Dressing	20
46-125	Medium Boot/Foot Dressing	20
46-126	Large Boot/Foot Dressing	20
46-133	Small Chest Dressing	20
46-134	Medium Chest Dressing	20
46-123	Large Chest Dressing	20
46-151	Large Burn Jacket	10
46-138	Small Buttocks Dressing	20
46-137	Medium Buttocks Dressing	20
46-130	Large Buttocks Dressing	15
46-131	Medium Hand Dressing	20
46-127	Elbow Dressing	20
46-128	Scalp Dressing	20
46-129	Face Dressing	20

Sofsorb Pads & Sheets

Product #	Description	Size	Qty/Cs
46-107	Permeable Pad	24" x 36"	20
46-108	Permeable Pad	33" x 36"	15
46-135	Permeable Sheet	36" x 72"	15
46-110	Non-Permeable Pad	33" x 36"	15
46-136	Non-Permeable Sheet	36" x 72"	15