


Product installation



Accessory 1:
1 main frame



Accessory 2:
1 backrest

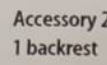

Accessory 2:
1 headrest



11. The backrest headrests are installed into the main frame in sequence (as shown in the picture)





12. Press and hold the switch in the picture to adjust the product 90°-120°

Auxiliary stand

Instruction Manual
Please read the manual carefully before use!

Product Size

(Error 1-3cm)

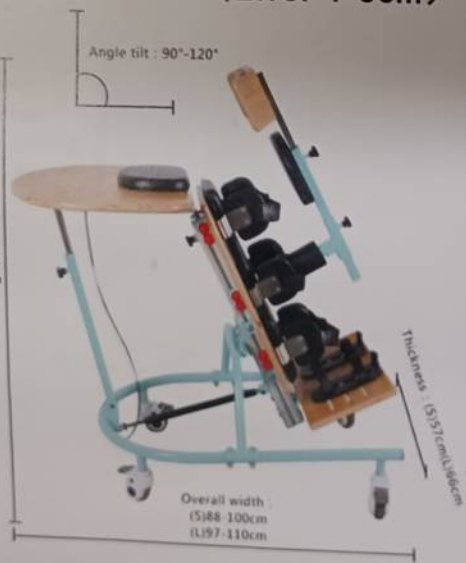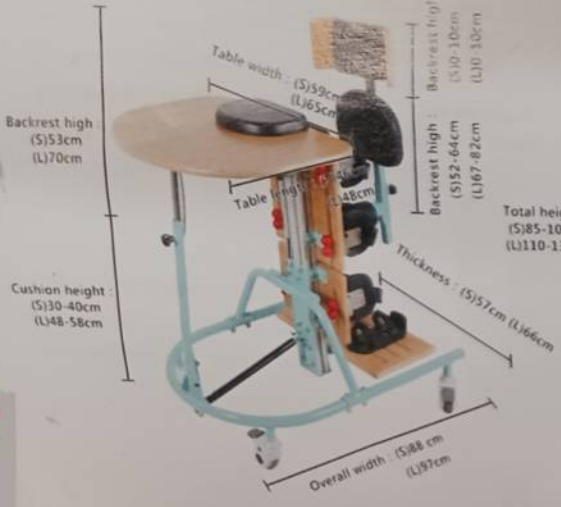

Raised toilet seat
L*W*H: (S) 88-100*57*85-105 (CM)
(L)97-110*66*110-138 (CM)
Error: 1-3 (CM) (0.4-1.18IN)

Box/Piece: 1 /1
Load bearing: (S)100(kg)
(L)150(kg)
Net weight: (S)25(kg)
(L)30(kg)

Service Hotline:1 (888) 714-8405

Product installation




1. Get all accessories out of the box




2. Caster locking brake to rotate the caster into the base hole



4. Four screws are fixed between the base of the main frame and the bracket, use a wrench to tighten the screws

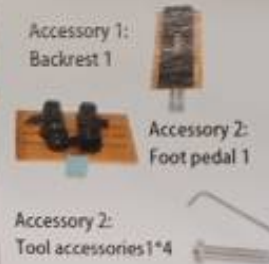

4. Install the foot plate into the bottom of the backrest frame (pay attention to the direction), adjust the nut hole position and then tighten the screw

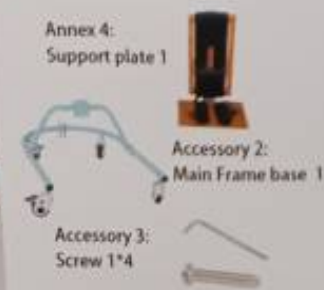

5. Align the backrest frame with the screw holes of the base bracket and then tighten the screws

Product installation



6. The telescopic rod is connected between the bottom bracket and the backrest bracket, and the height of the latch is inserted (pay attention to the direction as shown in the figure)



7. The table support rod is aligned with the hole of the main frame of the backrest. After inserting it, adjust the position and tighten the screws to fix it.



8. The bottom of the table board is connected with the base of the main frame, and the latch is fixed (pay attention to the direction as shown in the figure) before installing the cylinder switch under the table board



9. Install any set of baffles into the appropriate position and tighten the nut with the mounting screws (the other two sets are the same)



10. Install the seat cushion into the U-shaped groove of the back panel, adjust the height, and then tighten the screws