Use

1. Apply the brake to the chair!
2. Turn the foot rests away.
3. If necessary, fold the armrests up.
4. Support the less able person when they are taking a seat.
5. Shut the armrests and let the client's feet rest on the foot plates.


Stand up by carrying out the same procedure in reverse order.

Wheel blocking

1. Wheel blocking.
2. Unlocking.

Directional / braked wheel

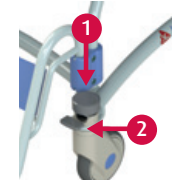
1. Wheel blocking.
2. Unlocking.

Foot rest

Both foot plates must be turned away for the less able person to take a seat in the chair. Make sure that the person places both feet on the foot plates when the chair is moving.

Armrest

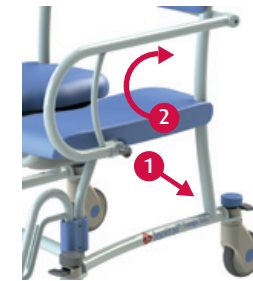1. Unlocking.
2. Turn backwards.

Close by following same process in reverse order.

Toilet function

- Remove the lid on the seat before the person sits down in the chair.
- *In combination with toilet:* The Tango XXL can be moved over almost every toilet.
- *With bedpan:* The Tango XXL can also be used as a toilet chair. Simply slide the stainless steel / plastic bedpan under the chair using the guides.

Cleaning and maintenance

The Tango XXL must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. Check the wheels regularly for hairs, dust and soap residues and remove when necessary.

Safety recommendations

- Read the Instructions for use carefully (found at: www.lopital.com) before use.
- Maximum load 300kg.
- Make sure that the brake is applied to the chair when transferring clients to and from the chair.
- Place both feet on the foot supports.

Important!

This instruction card may not be considered a replacement for the Instructions for use.

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