

Please read all of the following information before using this product.

TherAdapt[®] Products Inc. has designed and manufactured this product with the well being of the user and caregiver(s) in mind. It is our endeavor to provide products which are attractive, comfortable, useful, highly adjustable, long lasting and safe. As when using any product, precautions must be taken.



WARNING: This product must only be used under the recommendation of a physician and / or licensed therapist and with adult supervision. Never leave user unattended. It is your responsibility to properly use and maintain your Flip Top Easel. Failure to follow USE and CARE instructions or ASSEMBLE the easel properly as outlined may result in bodily injury.

THE MANUFACTURER IS NOT RESPONSIBLE FOR INJURIES RESULTING FROM ANY CUSTOMIZATION OR MODIFICATION OF THE EQUIPMENT THAT WAS NOT COMPLETED BY THE MANUFACTURER.

USE: The Flip Top Easel is designed for a single purpose: as a freestanding easel. It is to be used in conjunction with a stable support surface including, but not limited to a desk or table. **No other use is permitted.**

CARE: Take care to inspect the ongoing condition of your Flip Top Easel. Verify that all components are in good condition. Contact the manufacturer for replacement or repair of necessary items. All components may be cleaned with mild soap and water, or a gentle sanitizing spray. **Do not** use abrasive cleaners. Wipe wood components dry after cleaning. Excess amounts of liquid on the wood may damage the finish. Always store indoors.

WARRANTY: The manufacturer provides a full one (1) year warranty, beginning the date shipped by the manufacturer. The warranty provides full replacement of all parts against breakage when used in compliance with the manufacturer’s USE and CARE instructions. Use or modification beyond the normal capacity or range of adaptability designed into the products will void the warranty. The foregoing is TherAdapt’s sole liability for consequential damages.

PLEASE COMPLETE AND RETAIN FOR YOUR RECORDS

Flip Top Easel FT-100

Serial Number: _____

Purchase date: _____

TherAdapt[®] Flip Top Easel (FT-100) Instructions

Packing List:

(1) Flip Top Easel with gray non-skid tape along bottom edge

No assembly required



Use the easel with the book ledge on the 45° angle for reading.

By reversing the easel position, a writing angle of 30° is provided. The bottom edges of the easel have a non-skid surface for stability.

