



STRIDE 3-IN-1 ROLLATOR

Lifetime Limited Warranty

If you have a question about your MOBB Healthcare device or this warranty, please contact an authorized MOBB Healthcare dealer.

MOBB Healthcare warrants to the original consumer purchaser for as long as they own their MOBB product that it will be free from defects in material and workmanship under normal use. This warranty is only applicable to products used in residential applications within the United States and Canada and is not transferable to subsequent owners. Should defects in material or workmanship exist during the applicable warranty period in any MOBB Healthcare Product, MOBB Healthcare will elect at its discretion to either repair or replace the defective material. This warranty covers only parts and materials of the products supplied by MOBB Healthcare. Not covered under this warranty are wear and tear items or abuse of the product in a manner that was not intended for the product to be used. This warranty shall become void if the product is in any way modified, improperly adjusted or abused.



weight capacity: 300 lbs

MOBB Healthcare Ltd.

116A Viceroy Rd. Unit 5-6
Concord, Ontario, Canada
L4K 2M1

MHRL108BK



990000000882

Features



Cell Phone Pouch

Handle & Push Down To Lock Brake.

Padded Back Rest

Integrated Cane Holder Upper Clip

Side to Side Folding

Adjustable Handle Height

Removable Travel Bag with Front Reflector

Front and Rear Red Reflectors

Adjustable Seat Height

8" Rear Wheels

10" Front Wheels

Integrated Cane Holder Bottom Tray

Side to Side Foldability and Quick Release Frame Disassembly for Ultimate Portability

Operating Instructions

Assembly & Setup

ALWAYS CONSULT WITH YOUR HOME HEALTH CARE PRODUCT RETAILER OR MOBILITY PROFESSIONAL FOR PROPER FITMENT AND OPERATION.

FRAME ASSEMBLY

1. Insert caster tube onto frame.



2. Push the push pin lock on inside of frame. Ensure there is an audible "click" from the push pin.

3. Close the quick release handle as shown. Repeat at all four caster tubes.

4. Press seat down so that the seat locks engage and the seat is secure. Ensure that you hear the "click" of the lock engaging. Refer to arrow on image below.



Lock is engaged under the seat.

Operating Instructions

Assembly & Setup

Handle Height Adjustment



Turn the knob (on both sides) counter clockwise to loosen. Remove the knob/pin. Raise or lower the handle height so user's elbows are roughly at 90 degrees. Re-insert and tighten.



Seat Height Adjustment

Turn the knob (on both sides) counter clockwise to loosen. Do not remove. Raise or lower the seat height so user's knees are roughly at 90 degrees. Re-tighten.

Folding

1. Lift the seat handle



2. Bring seat rails together. Engage safety latch.



Manimum seat to floor height:
18.5"/47cm
Manimum floor to handle height:
31"/79cm



Maximum seat to floor height: 22.5"/56cm
Maximum floor to handle height: 41"/104cm



Safety Instructions

- Weight capacity 300 lbs.
- Regularly check screws and fasteners and retighten if required.
- Please make sure the rollator is fully opened, seat is down and the folding bar under seat is locked in the down position.
- Test rollator for stability before applying full weight to hand grips.
- Test the brake lever before operating the rollator. When wheels are locked, lock bar on rear wheels should prevent wheels from turning. When locks are released, the wheels should turn freely.
- Use the rollator only on even and solid surfaces.
- Always lock the brake lever before sitting and do not use seat when unit is on an incline or uneven ground.

DO NOT MOVE ROLLATOR WHILE SEATED: This is a walking aid only and is not to be used as a transportation device.