



Kaye Products, Inc.

ADAPTIVE EQUIPMENT AND REHABILITATION PRODUCTS
535 DIMMOCKS MILL ROAD HILLSBOROUGH, NC 27278 USA
TEL (919) 732-6444 FAX (800) 685-5293

VERTICAL STANDER CASTER BASE, MODEL VS1-C

(effective 10/15/02)

These are the user instructions for the VS1-C Casters. All persons who set-up or adjust the product should first read these entire instructions. Please store these instructions for future reference.

USE

The VS1-C casters fit the VS1 and VS1+C Kaye Vertical Stander. The casters allow easy relocation of the stander while the stander is not in use. A 2" caster is used so that the caster can fit under a standard counter's toe-space.

FRONT CASTER INSTALLATION

Attach the casters with the wooden extensions to the front of the stander as follows:

1. If you are retrofitting the casters to the stander, remove the existing base extensions from the stander.
2. Install the caster extensions with the wheels down and the push-pins out.
3. Depress the push-pins and slide the extensions into the ends of the base until the pins catch in the front set of holes. Unless the stander is used at a counter, the extensions should be used in this extended position.
4. If the stander is used at a counter, depress the push-pins and slide the extensions back until the push-pins catch in the rear set of holes. Do not use the stander with the extensions in this retracted position unless the stander is at a counter.

REAR CASTER INSTALLATION

Two 7/16" wrenches, a regular screw driver, and a phillips screw driver are needed.

Refer to the diagram and attach the casters as follows:

1. Position the stander upside-down.
2. Remove the plugs and rubber stoppers (if present) from the ends of the base-tubes.
3. Remove the nuts and washers from the ends of the casters.

4. Install the casters on the inside of the tubes so that the bolts protrude through holes in the tubes.

5. Place a flat washer, a split washer and then a nut on each screw and securely tighten the nut.

LOCKING THE CASTERS

Each caster has a flange on the side. Rotate the flange to lock and unlock the caster. Always keep the casters locked unless you are intentionally relocating the unit.

MAINTENANCE AND CARE

Inspect the unit regularly. Tighten the hardware as necessary.

If the unit needs service or spare parts, contact Kaye Products, Inc. or the distributor from whom you purchased the unit.

If a problem is discovered that may impact the unit's function, immediately cease use and contact Kaye Products, Inc.

Do not expose the unit to rain or submerge it in water.

Use a slightly damp cloth to clean the unit, but wipe the dampness from the wood after cleaning it.

Avoid any undue stress to the unit while using, storing, or transporting it.

LIMITED WARRANTY

If an item proves defective within two years of the original purchase, we will provide replacement parts in order to correct that defect. Normal wear and tear is not covered by the warranty.

Kaye Products, Inc. makes no other warranty, expressed or implied, and does not warrant the product as being fit for a particular purpose. The purchaser, owner, and user assume all risk of personal and property injury due to the use of the equipment.

CAUTIONS

1. Do not adjust the unit while it is in use.
2. Before use, ensure that all hardware is securely tightened.

3. Do not use the unit if there are broken or missing parts.
4. Do not alter the unit or use it in any way other than described in these instructions.
5. Do not leave the user unattended.
6. Use qualified supervision.
7. Observe all of the cautions and size and weight limits for the Kaye Stenders.
8. Do not roll the unit while it is use.
9. Keep the casters locked unless you are intentionally relocating the unit.

QUESTIONS?

If you have any questions, comments, or requests, please contact Kaye Products, Inc.



Our authorized representative for the European Medical Device Directives is Advena, Ltd., PO Box 30, Leominster, Herefordshire, HR6 0ZQ, UK.

