

## **LATERAL KNEE SUPPORTS, VS1-K & 103-K**

(effective 10/15/02)

These are the user instructions for the VS1-K and 103-K Lateral Knee Supports. All persons who set-up, adjust, or use these units should first read these entire instructions. Please store these instructions for future reference.

### USE

The VS1-K fits the VS1 and VS1+C Vertical Standers. The 103-K fits the 102, 103, 104, and 106 Prone Standers. Both lateral knee supports are designed to limit the abduction of the legs and to help keep the user's lower body aligned.

### INSTALLATION

The lateral knee supports are designed to replace the lower cushion that comes with the stander.

To install the support:

1. Remove the wing nut or knob that holds the lower cushion in place.
2. Remove the lower cushion.
3. Install the knee support with its longer screw on top.
4. Use the wing nut or knob to secure the unit in place.

### ADJUSTMENT

To adjust the width between the side cushions:

1. Loosen the knobs on the back of the knee support assembly.
2. Reposition the side cushions.
3. Securely tighten the knobs.

### MAINTENANCE AND CARE

Inspect the unit regularly. Tighten the hardware as necessary.

If the unit needs service or spare parts, contact Kaye Products, Inc. or the distributor from whom you purchased the unit.

If a problem is discovered that may impact the unit's function, immediately cease use and contact Kaye Products, Inc.

Do not expose the unit to rain or submerge it in water.

Use a slightly damp cloth to clean the unit, but wipe the dampness from the wood after cleaning it.

Avoid any undue stress to the unit while using, storing, or transporting it.

### LIMITED WARRANTY

If an item proves defective within two years of the original purchase, we will provide replacement parts in order to correct that defect. Normal wear and tear is not covered by the warranty.

Kaye Products, Inc. makes no other warranty, expressed or implied, and does not warrant the unit as being fit for a particular purpose. The purchaser, owner, and user assume all risk of personal and property injury due to the use of the equipment.

### CAUTIONS

1. Do not adjust the unit while it is in use.
2. Before use, ensure that all hardware is securely tightened.
3. Do not use the unit if there are broken or missing parts.
4. Do not alter the unit or use it in any way other than described in these instructions.
5. Do not leave the user unattended.
6. Use qualified supervision.
7. Observe all of the cautions and size and weight limits for the Kaye Standers.

### QUESTIONS?

If you have any questions, comments, or requests, please contact Kaye Products, Inc.



Our authorized representative for the European Medical Device Directives is Advena, Ltd., PO Box 30, Leominster, Herefordshire, HR6 0ZQ, UK.

10/15/02 I updated Advena's address. AWH