



## **BestCare Wall Mounted Lift System - Customization Guide**

In order to best fit the wall mounted lift system to your room, it is necessary to measure two adjacent walls of the room where you want to install the system.

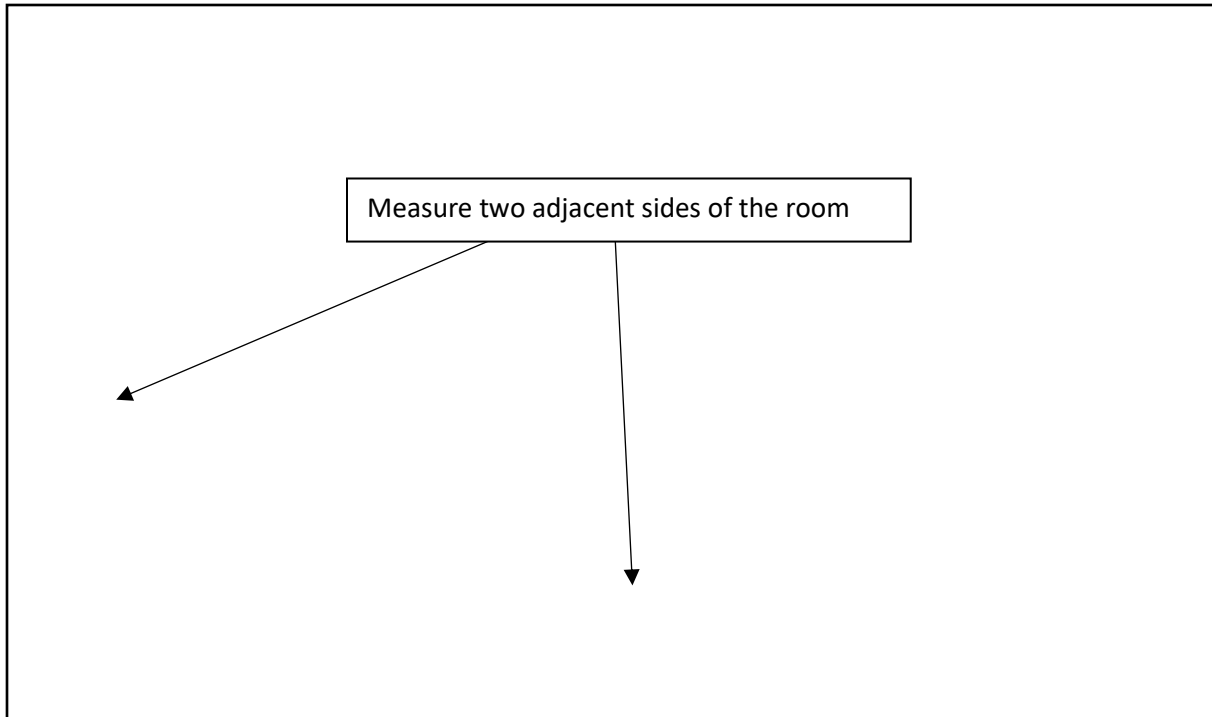

Based on these measurements, you will be able to select the best fitting rails for your room. Maximum coverage available is 23 ft by 23 ft and all setups must be rectangular/square. Minimum coverage is 14.1 ft by 9.02 ft.

Lift setup is comprised of two types of rails. One type of rail is the two rails that are mounted to the wall on opposite sides; the other type of rail is a traverse beam that goes across the room and connects to both wall rails.

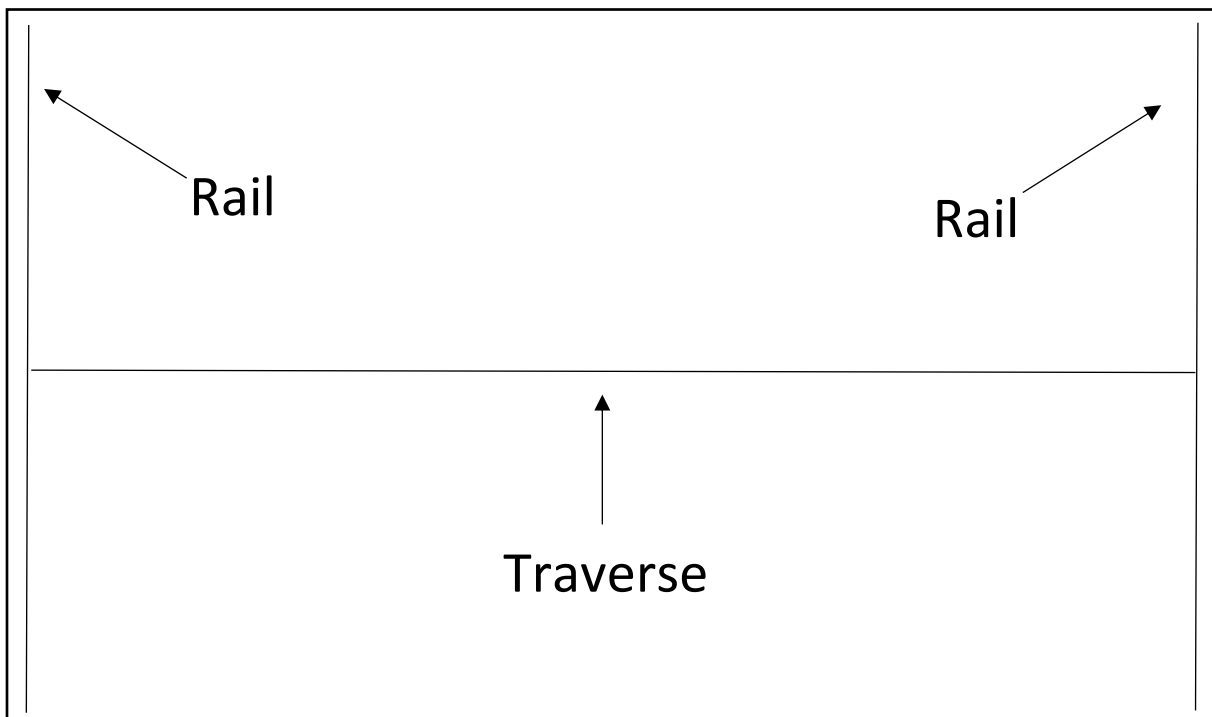

All Rail Track Length Options:

Track	Length
Etrack 2.75m	9.02 ft
Etrack 4.3m	14.1 ft
Etrack 5.5m	16.4 ft
Etrack 7m	23 ft

All Traverse Track Length Options:

Track	Length
135mm 4.3m	14.1 ft
135mm 5.5m	18 ft
135mm 7m	23 ft
135mm Reinforced 4.3m	14.1 ft
135mm Reinforced 5.5m	18 ft
135mm Reinforced 7m	23 ft

Based on these measurements select the best fitting tracks and traverse. As long as the setup remains the same with one of the standard setups and the only thing that is being change is track length, you should select all of the parts of the setup besides the tracks and traverse, and select the new tracks and traverse that you measured for in the **Additional Options** section in **Buying Options**. It is more cost effective to have the traverse run along the longer wall of the room, and the two rails run along the shorter wall.

Example Setup:

When configuring a room with the lengths of 18 feet by 17 feet you would select the 18 ft 135 mm traverse track for the 18-foot wall, and for the shorter 17-foot wall you would select the 16.4 ft Etrack. With this in mind you would then select the Buying Option that you are interested in and select all the accessories besides the tracks. So, for example if you were looking for a bedroom setup you would select the Bedroom Wall Mounted Lift System and select all of these accessories.

- ✓ Bestcare Luna 275/Bayonet [SKU#: BC-TL8-03028]
- ✓ Pendant/1320mm/ bottom exit [SKU#: BC-TL8-03101]
- ✓ Charger/holder/ Luna handset [SKU#: BC-TL3-00702]
- ✓ BestLift Luna Sbar/strap/275 kg [SKU#: BC-TL8-01968]
- ✓ ETrack End Stop [SKU#: BC-TL6-01133(2)]
- ✓ ETrack End Covers [SKU#: BC-TL6-01123]
- ✓ 135mm End Stops [SKU#: BC-TL6-01129(2)]
- ✓ 135mm End Covers [SKU#: BC-TL6-01190]
- ✓ Fixed Cart for 135mm Traverse Rail [SKU#: BC-TL6-02067]
- ✓ Adjustable Cart for 135mm Traverse Rail [SKU#: BC-TL6-02067]
- ✓ Cart for 135mm Track Bayonet [SKU#: BC-TL6-02003]

Leaving out:

- ✗ ETrack 2.75m [SKU#: BC-TL6-01029S(2)]
- ✗ ETrack Cover Plate [SKU#: BC-TL6-01032S(2)]
- ✗ 135mm 4.3m Traverse Track [SKU#: BC-TL6-01002]

After selecting your preferred room track setup go to the **Additional Options** section and select the following options.

- ✓ Etrack 5.5 M [SKU#: BC- TL6-01029(2)]
- ✓ Etrack Cover Plate 5.5 M [SKU#: BC- TL6-01032(2)]
- ✓ 135 mm 4.3m Traverse Track [SKU#: BC-TL6-01002]

Etrack Cover Plates are required for proper operation regardless of rail length