POSITIONING: Head - Lateral - Anterior - Hip/Pelvic - Abductor/Adductor - Knee - Foot - Replacement Parts Vest/Belt Attaching & Chest Strap



Trays

77-98

Tray Attaching 99-103

Sip-N-Puff /Joystick 104-107

Patient Transfer 108-111


Therapy 112-115

Strap Positioning Guides

• Secure strap positioning guide directly to chair or use with cam lock buckle or metal attaching clip

Strap Positioning Guides, Pair51030

KEEP STRAPS FROM FALLING OFF SHOULDERS

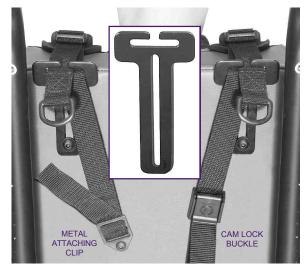

Hinged Collar

- No chair frame penetration
- One size fits 7/8", 1", and 11/8" tubing
- Use as mounting points for a vest or belt
- NOTE: Do not overtighten! When tightening, the two halves DO NOT TOUCH.





For our full line of Chest Straps see pg.40

FINGER HOLES



Chest Strap

- Adjustable strap with hook and loop closure
- Finger holes allow the user to easily manipulate the strap with limited hand use

Chest Strap

X-Small, 24" - 30"	
Small, 31" - 36"	
Medium, 38" - 48"	
Large, 50" - 60"	



Independent Living 116-121


HIP BELTS & CHEST STRAPS

Parts included vary depending on which belt you have. You may not have all parts shown here.


METAL ATTACHING CLIPS

Depending on width of strap, thread strap as shown.

Metal Attaching Clip & 1" Or 1.5" Strap

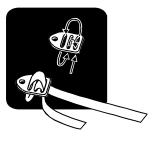


Metal Attaching Clip & 2" Strap

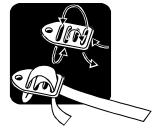


CAM LOCK BUCKLES

Thread strap through buckle and push top down to hold strap in place.

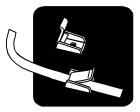


SLIDING DOUBLE-D CLIPS

Use Sliding Double-D Clips to loop straps around chair frame.

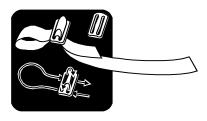

Use Sliding Double-D Clips with Metal Attaching Clips.

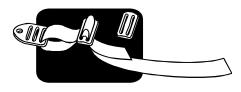

HINGED COLLARS

Attach Hinged Collars to chair frame. Attach Metal Attaching Clips or Cam Lock Buckles to Hinged Collars.



08756

