





Introducing Zeen!

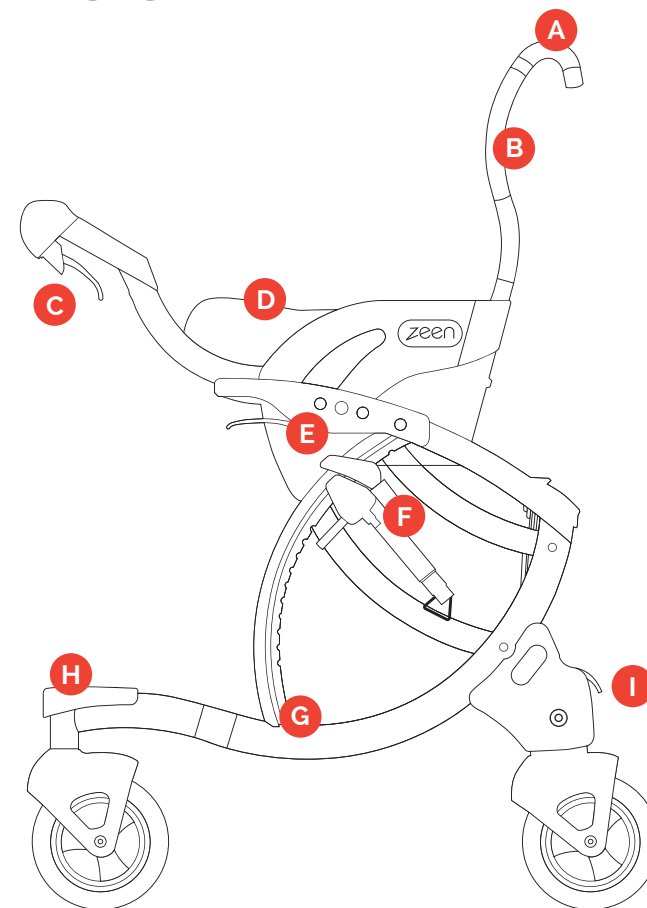

- A** Push handle
- B** Contoured backrest
- C** Coasting & Parking brake
- D** Memory foam seat
- E** Armrest with seat release levers
- F** Assisted lift
- G** Lightweight, folding aluminum frame
- H** Footrest for Barstool Mode
- I** Caster lock

Please be sure to read all safety instruction contained within the product manual.

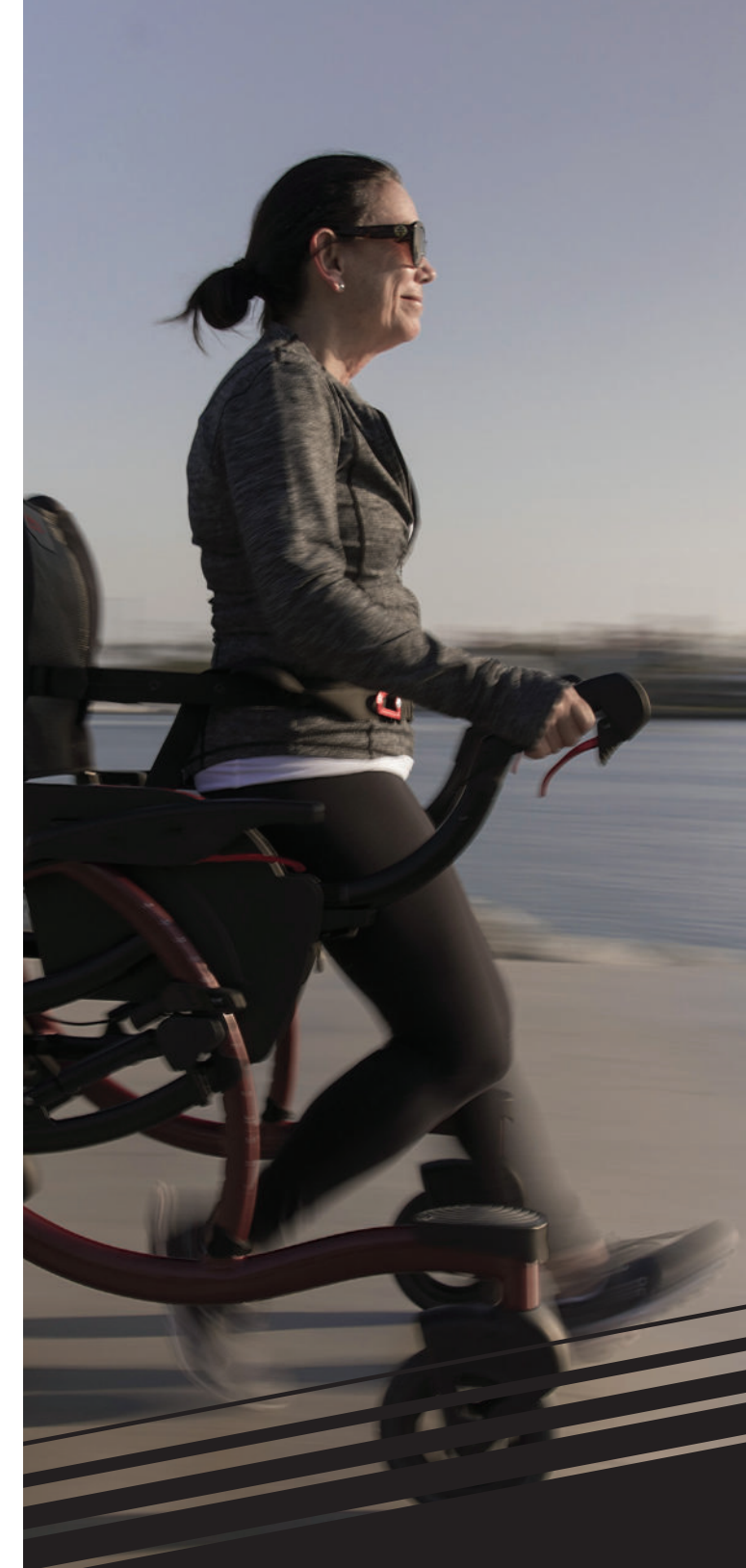

zeen®

Customer Support

📞 833-367-9336
 ✉ info@gozeen.com
 🌐 gozeen.com


Sit. Stand. Zeen.

A mobility device designed to bring more joy to life.

Unboxing and Setup Guide


gozeen.com

1

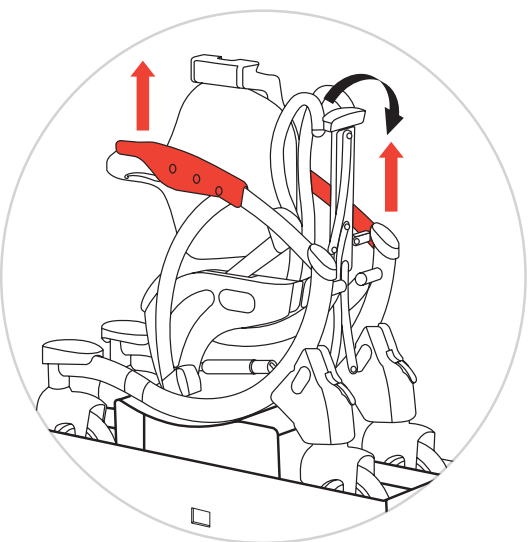


Team-lift the Zeen out of the lower packaging tray by gripping frame and armrests. (If alone, tilt both the Zeen and tray to the rear until the Zeen lies on its back; and slide off the tray forward. Then carefully tilt the Zeen back up on its wheels.)

2

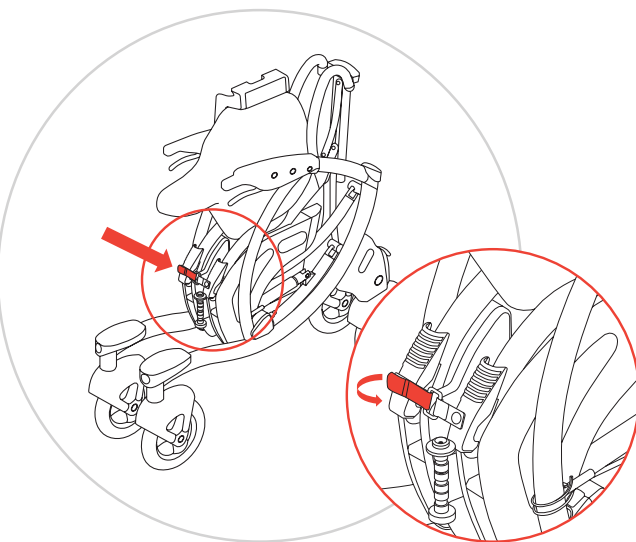


With the Zeen upright and wheels on the ground, remove protective plastic wrap. Then pull open the front Velcro folding strap. Rotate the straps vertical for easy use of the handlebars.

3

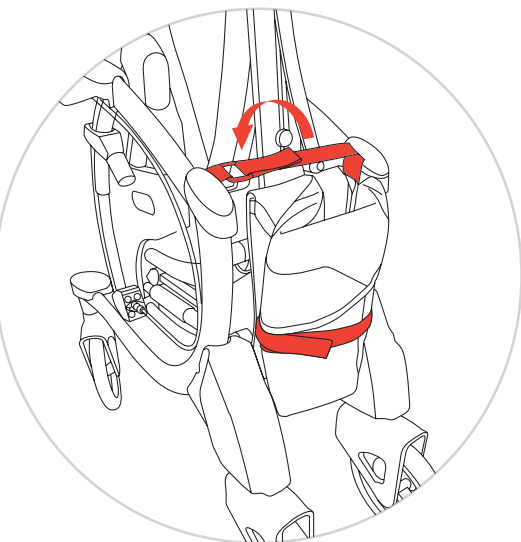

Pull open rear folding strap and rear bag strap.

4

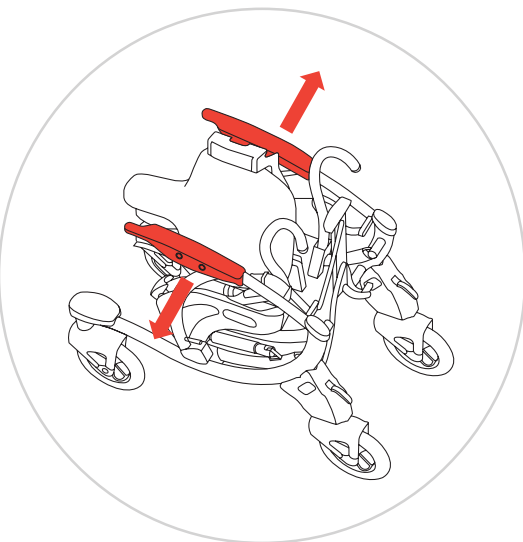

Unfold the Zeen by gripping and expanding the two side frame tubes near arm-rests. When mid-way expanded, pivot handlebars forward.


⚠ WARNING

Follow instructions in this Guide and the Product Manual carefully.

- Properly adjust and use seatbelt at all times to prevent falling from the Zeen.
- Ensure children are clear when adjusting seat height to prevent them from getting caught in the seat mechanisms.
- Use caution on slopes and when used in bar stool mode to prevent tipping and falling.
- Maintain control and awareness of surroundings when coasting to avoid collisions and tipping.


Improper setup, use, or maintenance of Zeen could lead to serious injury or death.

5A

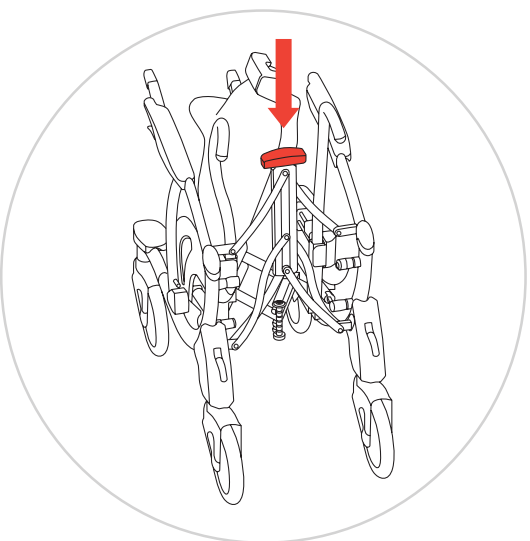

Push down on the red rear folding handle and lock into expanded position.


5B

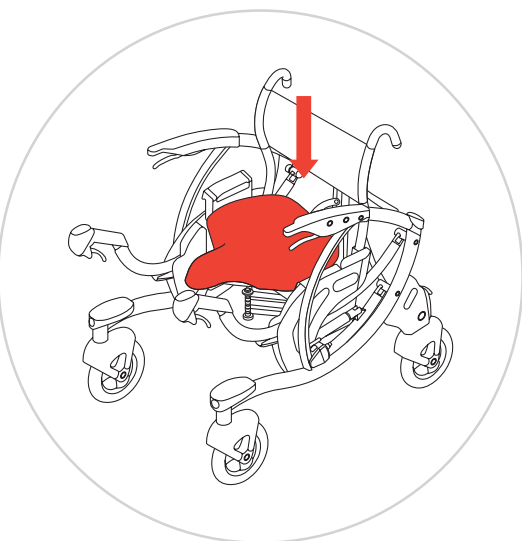

Next push down on seat until audible click confirms the seat is fully locked.


⚠ Handlebars must be fully deployed to lock seat in place. Cross-frame handle and seat must be locked properly in place. Failure to do so may result in instability and tipping during use.

6

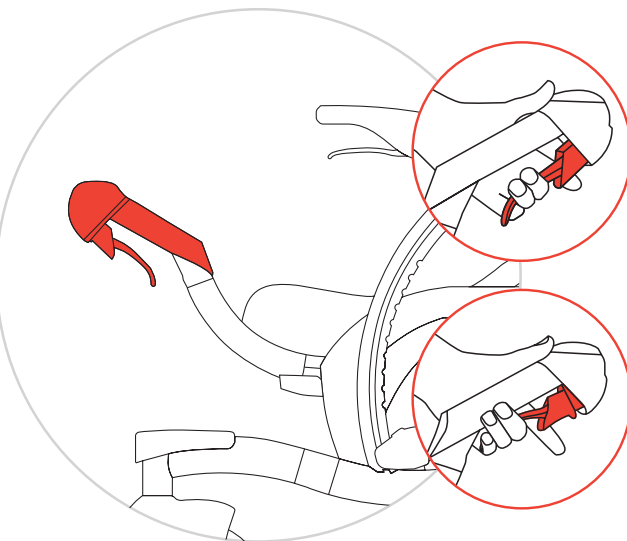

On the handlebars, fully squeeze the long coasting brake levers with middle, ring and little fingers, and add index finger (or your other hand) to pull up the short parking latches until they 'park' the brakes. Sit down on the Zeen seat.

7

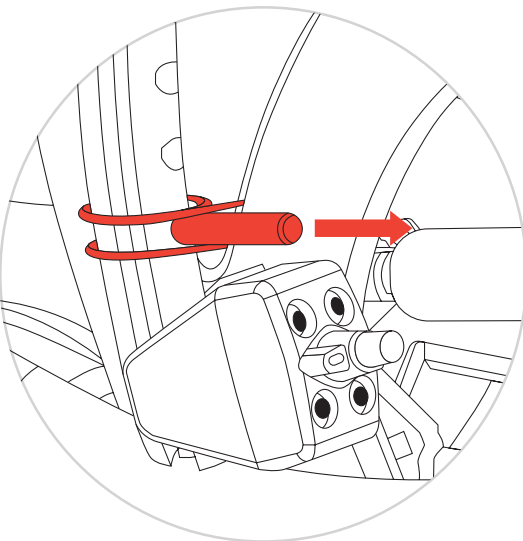

Remove the shipping pins from the frame tubing on both sides by freeing the elastic cord and then pulling the pin from the hole in the strut. Please retain these shipping pins for future use when transporting the Zeen.

⚠ Failure to remove both pins may result in instability and tipping during use.

8



Please read all safety instructions in the product manual and watch training videos at gozeen.com.

Use QR code to access our Zeen training videos and frequently asked questions.



Visit gozeen.com for usage instructions.