

EasyStandⁱⁱ Zing

MPS, Supine, MPS Tilt Table (TT), Supine Tilt Table (TT), Vertical & Prone **size1**

Please ensure the Owner's Manual stays with the unit at all times.



Owner's Manual

SYMBOL LEGEND



CAUTION/ATTENTION



PINCHPOINT

SN

SERIAL NUMBER



Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

Afin de préserver l'environnement, veuillez confier ce produit à votre centre de recyclage à la fin de sa durée de vie.

Nachdem dieses Produkt ausgedient hat, entsorgen Sie es bitte auf umweltfreundliche Weise, indem Sie es zu der entsprechenden Recyclingstelle bringen.

Wees a.u.b. milieubewust en recycleer dit product via uw recycleerder na het verstrijken van de levensduur.

Sea responsable con respecto al medio ambiente y recicle este producto al finalizar su vida útil en el lugar de reciclaje correspondiente.

Siate responsabili verso l'ambiente e riciclate questo prodotto, al termine della sua vita d'uso, presso la vostra sede di riciclaggio.


Kierrätä tämä tuote sen käyttöiän lopussa ympäristövastuullisella tavalla paikallisella kierrätyslaitoksella.

Var miljömedveten och återvinn denna produkt efter dess användning.

Vi ber deg ta hensyn til miljøet og resyklere dette produktet ved hjelp av de rette resyklingsfasilitetene når produktet ikke lenger kan brukes.

Vis venligst hensyn til miljøet ved at levere produktet til det nærmeste genbrugsanlæg, når det ikke længere fungerer.

Por favor, assumo responsabilidade ambiental, reciclando este produto em suas instalações de reciclagem no final de sua vida útil.

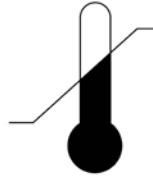

**ENVIRONMENTAL
CONDITIONS**



DATE OF MANUFACTURE



SEE INSTRUCTIONS FOR USE



MANUFACTURER

R_xonly

Table of Contents

Introduction	1
Safety Precautions	2
Assembly	3
Adjustments & Usage	11
Parts Detail	19
Maintenance	27
Limited Warranty	28

Date Purchased _____ Serial # _____

Introduction

Congratulations on your purchase of the EasyStand Zing standing frame. We at Altimate Medical have designed the EasyStand Zing with your needs in mind.

EasyStand Description

The EasyStand line of products is intended for individuals who are able to transfer with assistance from a caregiver and/or lifting device. It is a useful tool for rehabilitation, home, school, and work use. EasyStand standing frames support the user from various positions to the standing position. Various support and positioning options may be added to facilitate individual needs.


Indications for Use

The EasyStand Zing products have a flat surface that can be adjusted from the prone or supine position to increase tolerance to an upright standing position. The Zing Size 1 is indicated for most individuals weighing up to 70 lbs.

This manual is an integral part of your standing system and should be read completely and thoroughly by anyone that will be operating the unit. This will ensure the EasyStand Zing is assembled properly and used in the most beneficial manner.

The EasyStand Zing must be used with a qualified adult attendant present. Should any problem arise that you cannot solve by reading this manual, please contact your local supplier or call Altimate Medical, Inc., at 800-342-8968 or 507-697-6393.

It is necessary to inspect your EasyStand Zing at least weekly to ensure it is in safe operating condition. EasyStand adjustments that are listed in the product owner's manual can be made by the operator by following the instructions for use. After an adjustment is made, inspect the EasyStand to ensure all hardware and adjustable components are properly secured per the instructions for use. Replacement of worn parts, major adjustments, or any other important corrections should be handled by an authorized supplier or service center. Only Altimate Medical, Inc., approved replacement parts should be used to ensure safety and performance.

Do not use the EasyStand Zing without consulting your physician. Failure to consult a qualified physician prior to using the EasyStand Zing could lead to severe medical problems. We recommend a good standing program be established and supervised by a qualified physician or occupational/physical therapist. only




Join the Standing Community!

Our goal is simple - to empower people with disabilities to stand up for their health. Connect, share, and learn from other people with disabilities, special needs parents, clinical professionals, and the experts in standing on one of our social networking communities.

Read, share, and comment on the EasyStand Blog at blog.easystand.com
Find EasyStand on these social networking sites:



Safety Precautions

- Please read and understand all safety precautions and user instructions prior to use. Call Altimate Medical at 800-342-8968 or 507-697-6393 with any questions or for additional information.
 - If a user does not meet the specifications below and has not been properly fitted by a qualified therapist or physician, Altimate Medical, Inc., does not recommend using the EasyStand Zing.
Zing Size 1 - is designed to accommodate most individuals up to 44" (112 cm) and up to 70 lbs. (32 kg).
For more detailed specifications visit our website www.easystand.com
 - The EasyStand Zing should not be used without first consulting a qualified physician. Failure to consult a qualified physician prior to using the EasyStand Zing could lead to severe medical problems. We recommend a good standing program be established and supervised by a qualified, registered physical/occupational therapist.
 - The EasyStand Zing should always be used with a qualified adult attendant present. Consult with your therapist or physician for proper foot placement and angle. Never leave a child unattended in the Zing.
 - Do not make foot plate adjustments to the Zing while a child is standing.
 - It is necessary to inspect your EasyStand Zing at least weekly to ensure it is in safe operating condition. Pay particular attention for loose hardware. Replacement of worn parts, major adjustments, or any other important corrections should be handled by an authorized dealer or service center. Only Altimate Medical, Inc. approved replacement parts should be used to ensure safety and performance.
 - If your EasyStand is being used in the home environment, please inspect your unit prior to each use to ensure there is no damage or unexpected wear to the EasyStand that may have been caused unintentionally by pets, pests or children.
 - Ensure the necessary support and alignment options are in position prior to placing the user into the standing position.
 - Storage/Transport Conditions: Temperature: 14°F -122°F (-10°C - 50°C), Relative humidity: 20% - 90% at 86°F (30°C), Atmospheric Pressure: 700 to 1060hPa.
 - If the unit was exposed to temperatures of less than 50°F (23.5°C), allow unit to warm to room temperature before use.
 - The expected service life of an EasyStand is considered 5 years under normal use conditions. Note: This may vary based on frequency of use.
 - Standing equipment is designed for indoor use on level surfaces only. Keep casters locked while in use.
 - It is not recommended to transport the unit with a user in the standing position.
 - Extreme movements laterally or front to back may cause the unit to tip.
 - It is recommended the equipment be stored at normal room temperature and humidity levels.
 - Altimate Medical products are specifically designed to be used with Altimate Medical accessories and options. Unless otherwise noted, accessories and options from other manufacturers have not been tested by Altimate Medical and are not recommended for use with Altimate Medical products.
 - Product Modifications made without express written consent (including, but not limited to, modification through the use of unauthorized parts or attachments) are not recommended and will void the product warranty.
-  **Caution:** This equipment is designed to accommodate a range of users; therefore the unit has various adjustment slots and holes. Avoid placing fingers or other body parts in any of these areas.
-  **Caution:** While in use this equipment has various moving parts, familiarize yourself with the unit and avoid placing hands or fingers near any of these areas.
-  **Caution:** Advise bystanders to keep clear of the unit while in use.


Assembly

1. Remove everything out of the box and place the mast onto the floor as shown.

Remove all plastic wrap.



2. If the gas spring lockout was ordered, it will be shipped in the locked position. Pull the lock out and turn to unlock.

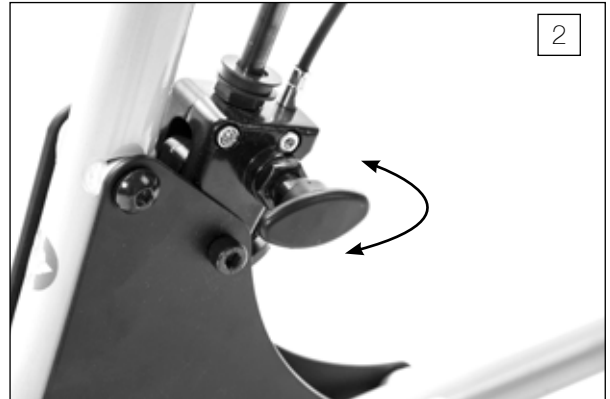

3. For the Supine, Vertical and Prone units, place unit up on end like shown in step 5, for the MPS and TT, depress the foot pedal and allow the mast post to come up to a vertical position.



4. Remove the bag of hardware from the wheelbase. It contains the 4 bolts for assembly.



5. Turn the wheelbase over and line up the short end with the foot pedal.




6. Start all 4 bolts. Do not tighten at this time.




7. Once all 4 bolts have been started, tighten them securely. Place unit onto its wheels and lock casters.



8. If the multi-adjustable knee pads or calf pads were ordered, loosen knobs on clamps and move up into position.

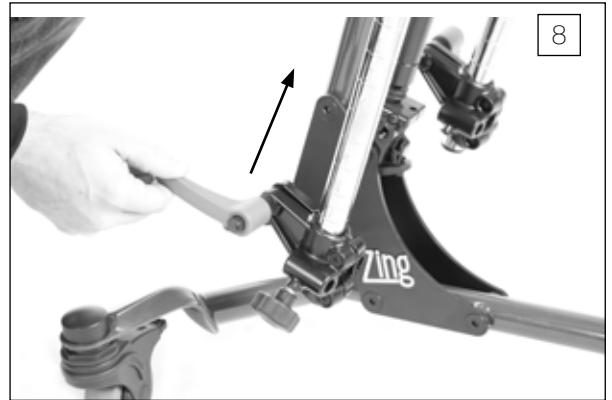

9. For the Prone and Supine units depress the foot pedal to allow the legs to go past 90°.

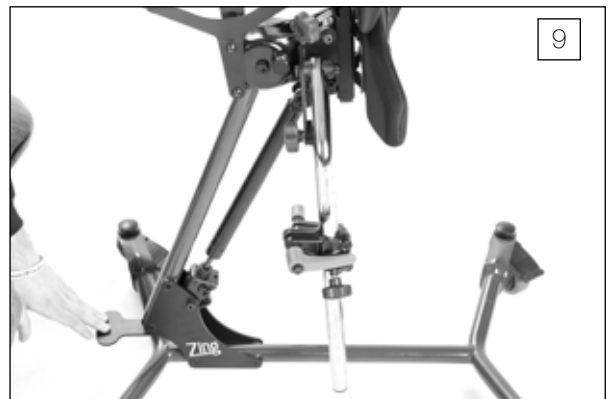

10. Tighten the lug on the side of the mast securely.



11. If the calf pad with knee strap was ordered, place the assembly onto the legs as shown.



12. Once the calf pad with knee strap is in position, tighten knobs securely.



13. If the independent adjustable calf pads were ordered, loosen the ratchet handles on the clamps and position as shown. Insert the calf pads into the inside of the clamp. Tighten securely.



14. Insert multi-adjustable knee pads into the outside of the clamp. Tighten securely.



15. If the platform foot plate was ordered, place the assembly onto the legs as shown. Tighten securely.




16. If the multi-adjustable foot plates were ordered, place each foot plate onto the legs with the handles facing out. Tighten knobs securely.



17. If the hip supports were ordered, insert hip supports into the bracket behind the pelvic pad. Tighten securely.



18. If the lateral supports were ordered, insert lateral supports into the bracket behind the upper body support pad. Tighten securely.

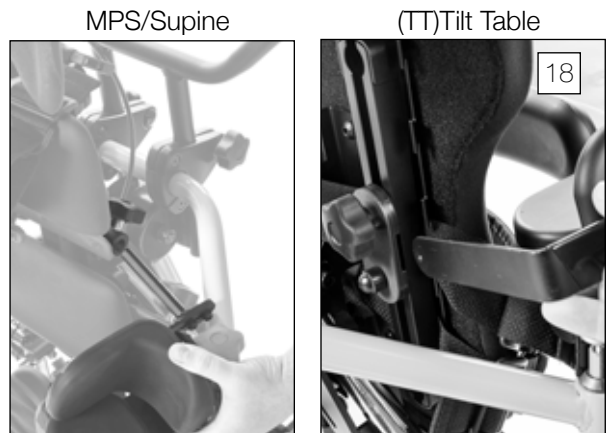

MPS, Supine or Tilt Table

19A. If the black molded tray was ordered, insert the two tray arms into the tray bracket as shown.

19B. Flip the gray plunger back into the locked position and tighten knob securely.



MPS, Supine or Tilt Table

20A. If the multi-adjustable tray was ordered, there are two screw knobs in the parts bag. Turn the screw knobs slightly into the tray bracket on the unit.

20B. Loosen the adjustment knob under the tray, extend the tray and place the tray bracket into the base. Tighten securely.

Note: Make sure washer is in place.



Washer

Vertical or Prone

21A. Remove 4 bolts from bottom of the tray, set bolts aside.

21B. Place tray onto bracket as shown.



Vertical or Prone

22. Start 2 bolts closest to the pads, then start front 2 bolts. Once all bolts have been started, tighten securely.



Vertical or Prone

23A. If the optional push handle was ordered, remove the bolts from the bracket as shown.



23B. Tilt one side of the push handle down to line up with the hole in the mast. Insert bolt and place nut onto bolt. Tighten securely. Repeat steps for the other handle.



24A. If the Zing Tilt Table (TT) was ordered, place the bracket for the planar head support onto the upper body support pad, making sure the spacers are in between the bracket and pad.



21B. Tighten bolts securely.




Adjustments & Usage

Trays

Black Molded/Multi-Adjustable Tray Height

25. To adjust tray height, loosen knobs on the back of unit and set to desired position. Tighten securely.

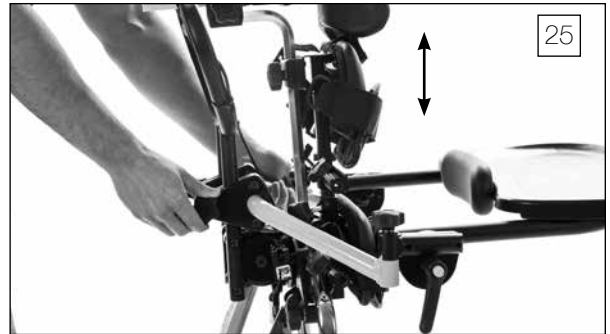


Black Molded Tray Depth

26. To adjust the tray, loosen the knob until resistance is felt, then flip the gray lever in the center of the knob fully open.


⚠ DO NOT continue to turn the knob counterclockwise. Forcing the knob past its stop will damage the mechanism.

The tray may now be removed completely, or adjusted to the desired position using the click detent positions as a guide.

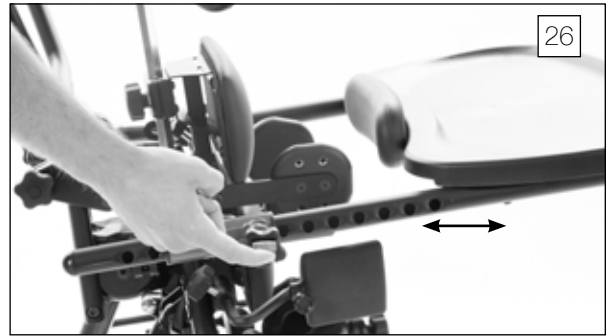


Multi-Adjustable Tray Depth

27. Loosen knob under the tray, pull or push into desired position. Tighten securely.

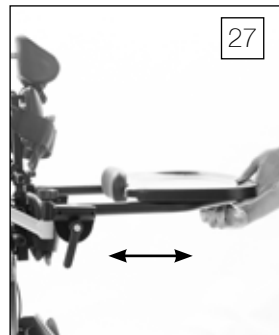

Multi-Adjustable Tray Angle

28. To adjust tray angle, loosen ratchet handles on both sides of the tray, set to desired position. Tighten securely.

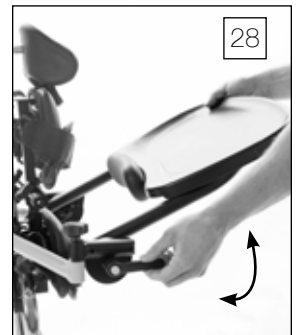


Multi-Adjustable Tray Swing-Away

29. Loosen knob on one side only until you are able to swing tray away.



30. Swing tray away to allow for transfers.

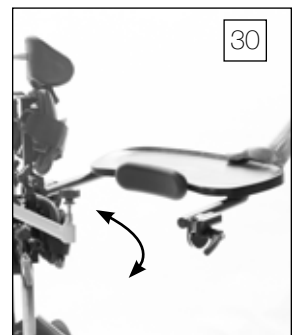


Hip Supports

31. The hip support width can be adjusted by loosening the knobs behind the pelvic pad. Hip supports come with both thumb knobs and set screws for fore/aft positioning. Tighten securely.




Lateral Supports (MPS, SUPINE, VERTICAL, AND PRONE)

32. The lateral support width can be adjusted by loosening the knobs on the back of the unit. Always tighten securely. The depth can be adjusted by loosening the two bolts, once the desired position is reached. Tighten securely.

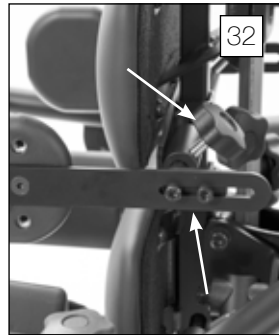

33. The lateral supports have a number of attachment holes on the back of the pad which will allow for additional positions vertically and horizontally. The pads may also be bolted at a high, low, or neutral position in order to effect greater pad range.

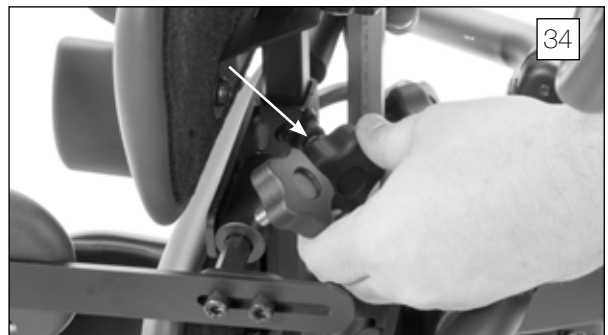

34. To adjust the lateral support height, loosen knob, adjust to desired position. Tighten securely.

Lateral Supports (TILT TABLE)

35. To adjust the lateral support height, loosen knob, adjust to desired position. Tighten securely.




Calf Pad with Knee Strap

36. To adjust the calf pad with knee strap, loosen knobs on each side. Position fore/aft and up/down to the desired setting. Tighten securely.




Multi-Adjustable Knee Pads

37. To adjust the multi-adjustable knee pad height, loosen the ratchet handle. Adjust to desired position. Tighten securely.

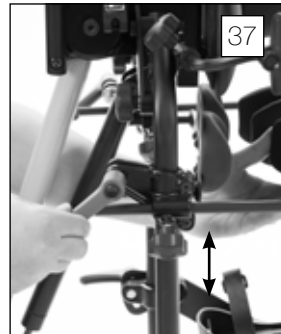

38. To adjust the multi-adjustable knee pad depth, loosen the knob. Adjust to desired position. Tighten securely.

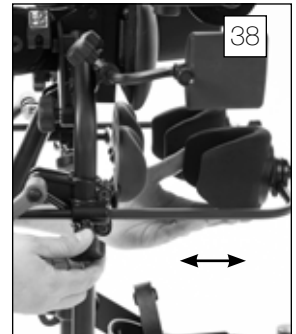

39. To adjust the multi-adjustable knee pad position, loosen knob and rotate to desired position. Tighten securely.




Independent Adjustable Calf Pads

40. To adjust the independent adjustable calf pad depth, loosen the knob. Adjust to desired position. Tighten securely.



41. Remove bolts and position as desired to adjust calf pad height. Tighten securely.

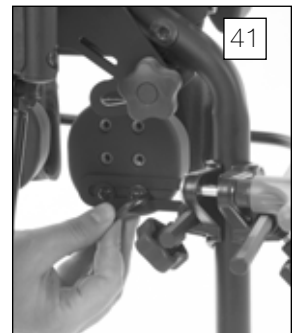

Head Support


42. To adjust the head support height, loosen the knob on the back, move to desired position. Tighten securely.

43. To adjust the head support depth, loosen the knob on the back of the headrest and slide into desired position. Tighten securely.

To adjust the head support angle, loosen the ratchet handle and place into desired position. Tighten securely.


Upper Body Support Pad

44. To adjust the upper body support pad height, loosen knob, depress spring button and adjust to desired position. Tighten securely.

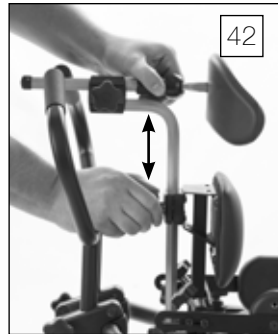


Platform Foot Plates

45. **Height**-Platform Foot Plates are only height adjustable. Loosen knob, set to desired position. Tighten securely.

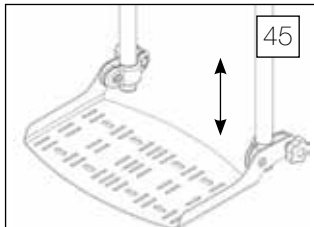


46A. **Straps**-Position straps into desired slots.

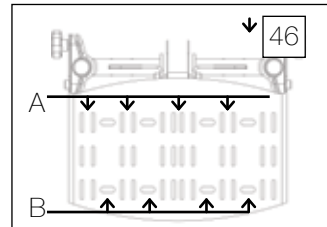


46B. **Foot Holders**-Position foot holders into desired position. Insert bolt and tighten knob securely.

Multi-Adjustable Foot Plates


47. **Height**-Loosen the ratchet handles connecting the foot bracket to vertical leg tubes to adjust foot plate height.

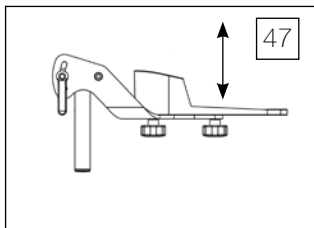


48. **Plantar/Dorsi**-Loosen ratchet handles to tilt foot plates. Tighten securely.

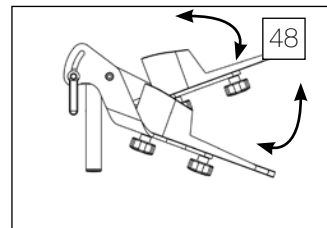


49. **Abduction**-Loosen knobs to slide foot plates together. Tighten knobs securely

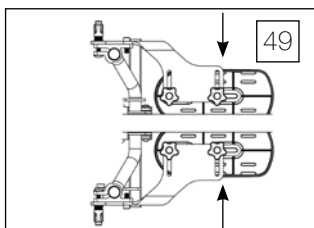


50. **Forward/Aft**-Loosen knobs underneath foot plates, slide foot plates into position. Tighten knobs securely

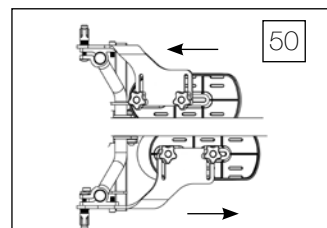

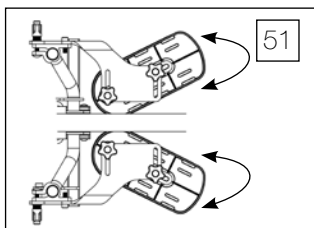

51. **Toe-in/Toe-out**-Loosen knobs underneath foot plates, slide foot plates into position. Tighten knobs securely

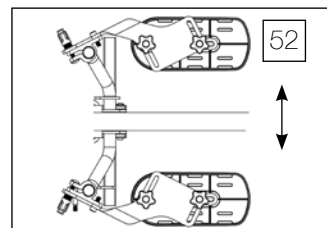

52. **Abduction**-Loosen ratchet handles to spread apart foot plates, tighten securely. Loosen knobs under foot plates and slide foot plates apart. Tighten knobs securely



 **Caution:** Never reposition the foot plates with the child in the standing position.


 **Caution:** Avoid placing fingers in the adjustment slot.

Leg Abduction

53. Loosen knob behind the pelvic pad.




54. Abduct each leg out to the desired position up to 30 degrees per leg. Tighten securely.

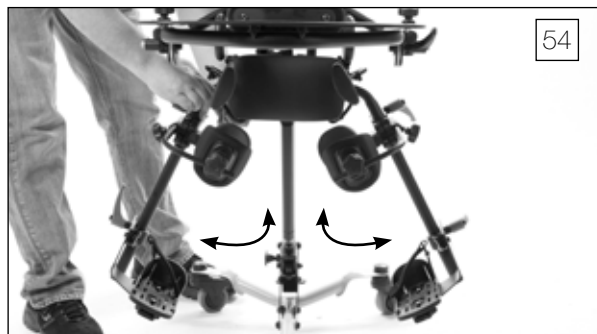

Supine (MPS only)

55. Flip the lever located behind the pelvic pad to the “supine to stand” position.

56. Adjust to the desired supine position by depressing the foot pedal and tilting. This can be done with the user in the unit.

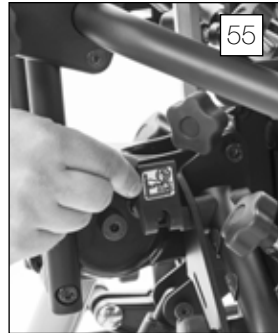

Prone (MPS only)

57. Flip the lever located behind the pad to the “prone” position.


58. Adjust to the desired prone position by depressing the foot pedal and tilting. This can be done with the user in the unit.



Dual Control

59. Squeeze the hand lever to place the unit in the desired position.


60. Use the foot pedal to place the unit in the desired position.

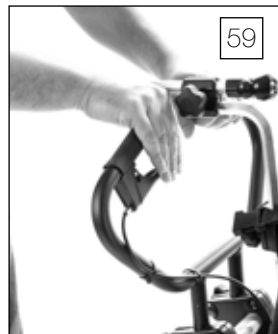

Gas Spring Lockout

61. Pull the lockout knob out and twist to unlock. Your unit can now be positioned using the foot pedal or hand lever.

62. Twist until the knob is vertical and locks into position. The gas spring is now locked from unintended movement.

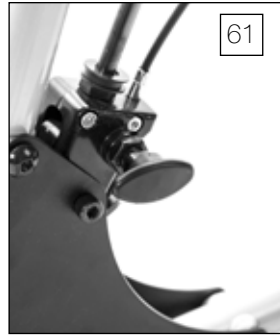

Directional Locking Caster

63. Lock the right rear caster to allow the unit to guide straight. Locking the directional caster locks the directional position of the caster. This will not prevent the caster from rolling.




Face Aperture

64. Loosen knob on pad. Insert face aperture into the slot. Tighten knob securely.




Parts Detail-Supine

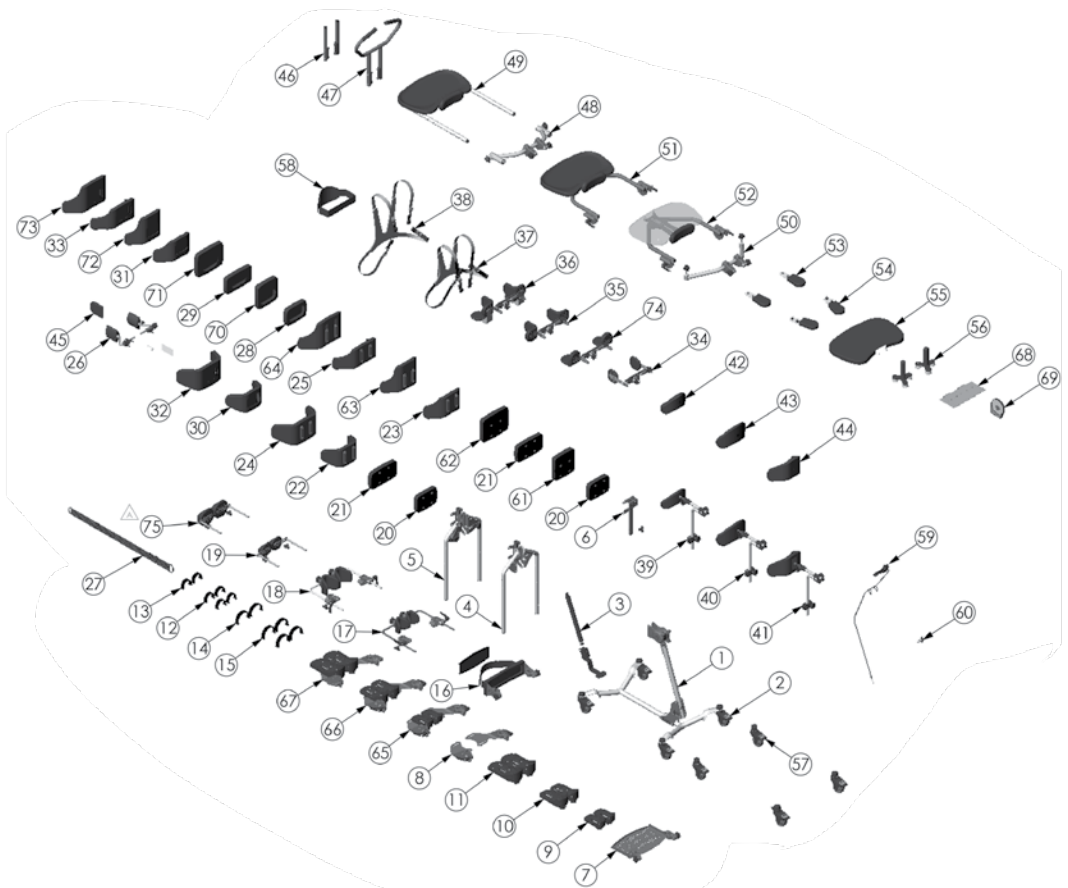


ITEM NO.	PART NO.	REV	QTY.	NAME	ITEM NO.	PART NO.	REV	QTY.	NAME
1	A5522	A	1	ASMBLY BASE ZING SUPINE	38	T50088	000	1	ASMBLY CHEST VEST SM
2	A5526	000	1	ASMBLY CASTERS BASIC MPS	39	Y5626	A	1	ASMBLY HEAD SUPPORT XS
3	A5528	000	1	ASMBLY GAS SPRING LIFT MPS	40	Y5628	A	1	ASMBLY HEAD SUPPORT SMALL
4	A5530	B	1	ASMBLY LEGS MPS STANDARD	41	A5586	000	1	ASMBLY HEAD REST CONTOUR SM
5	A5532	B	1	ASMBLY LEGS MPS ABDUCTION	42	A3035	000	1	SLIP COVER HEAD REST XS
6	A5534	000	1	ASMBLY MAST MPS	43	A3036	000	1	SLIP COVER HEAD REST SM
7	A5534	000	1	ASMBLY PLATFORM FOOT PLATE MPS	44	A3037	000	1	COVER HEAD REST CONTOUR SM
8	A5538	000	1	ASMBLY FOOT PLATE MPS MULTI-ADJ	45	A3039	000	1	SLIP COVER 3.5" LATERAL PAD
9	A5540	000	1	ASMBLY FOOT PLATE MPS EI	46	A5592	000	1	ASMBLY TRAY ASSEMBLY MOUNT MPS
10	A5542	000	1	ASMBLY FOOT PLATES XS	47	A5594	000	1	ASMBLY PUSH HANDLE MPS
11	A5544	000	1	ASMBLY FOOT PLATES SM	48	A5595	000	1	ASMBLY TABLE MOUNT BASIC MPS
12	A5546	000	1	ASMBLY SECURE FOOT STRAPS XS	49	A5596	000	1	ASMBLY TRAY MOLDED MPS
13	A5548	000	1	ASMBLY FOOT STRAPS XS	50	A5599	A	1	ASMBLY TABLE MOUNT TILT MPS
14	T50080	A	1	ASMBLY FOOT STRAP PED	51	A5600	A	1	ASMBLY TRAY SWING AWAY MPS
15	T50082	A	1	ASMBLY FOOT STRAP SECURE PED	52	A5602	A	1	ASMBLY TRAY SWING AWAY CLEAR MPS
16	A5550	000	1	ASMBLY KNEE PAD SOFT MPS	53	A5604	000	1	ASMBLY FOREARM WINGS MPS
17	A5556	000	1	ASMBLY KNEE MULTI-ADJUSTABLE XS	54	A5606	000	1	ASMBLY FOREARM WINGS CLEAR MPS
18	A5558	000	1	ASMBLY KNEE MULTI-ADJUSTABLE SM	55	A3038	000	1	PADDED SLIP COVER TRAY MPS
19	A5560	000	1	ASMBLY CALF PAD INDEPENDENT MPS	56	NG50042	000	1	SUPPORT GRIP OPTION
20	A5562	000	1	ASMBLY PAD PLANAR 5" X 7"	57	A5608	000	1	ASMBLY CASTERS DIRECTIONAL MPS
21	A5564	000	1	ASMBLY PAD PLANAR 5" X 9"	58	A5610	000	1	ASMBLY CHEST STRAP MPS
22	A5566	000	1	ASMBLY CONTOUR PELVIC 5" XS	59	A5612	000	1	ASMBLY DUAL LIFT CONTROL MPS XS
23	A5568	000	1	ASMBLY CONTOUR BACK 5" XS	60	A5614	000	1	ASMBLY GAS SPRING LOCK OUT
24	A5570	000	1	ASMBLY CONTOUR PELVIC 5" SM	61	A5616	000	1	ASMBLY PAD PLANAR 7" X 7"
25	A5572	000	1	ASMBLY CONTOUR BACK 5" SM	62	A5618	000	1	ASMBLY PAD PLANAR 7" X 9"
26	A5574	000	1	ASMBLY HIP GUIDES MPS XS	63	A5620	000	1	ASMBLY CONTOUR BACK 7" XS
27	A3011	000	1	POSITIONING BELT ADJUSTABLE XS	64	A5622	000	1	ASMBLY CONTOUR BACK 7" SM
28	A3028	000	1	SLIP COVER PLANAR 5" X 7"	65	A5624	000	1	MULTI-ADJUSTABLE FOOT PLATE MPS EI
29	A3029	000	1	SLIP COVER PLANAR 5" X 9"	66	A5626	000	1	MULTI-ADJUSTABLE FOOT PLATE MPS XS
30	A3030	000	1	SLIP COVER CONTOUR PELVIC 5" XS	67	A5628	000	1	MULTI-ADJUSTABLE FOOT PLATE MPS SM
31	A3031	000	1	SLIP COVER CONTOUR BACK 5" XS	68	80809	A	1	TOOL POUCH
32	A3032	000	1	SLIP COVER CONTOUR PELVIC 5" SM	69	80246	000	1	SEAT ANGLE LOCATOR
33	A3033	000	1	SLIP COVER CONTOUR BACK 5" SM	70	A3045	000	1	SLIP COVER PLANAR 7" X 7"
34	A5576	A	1	ASMBLY LATERAL GUIDES MPS SM	71	A3046	000	1	SLIP COVER PLANAR 7" X 9"
35	A5578	A	1	ASMBLY LATERAL W/ ELBOW PAD MPS XS	72	A3047	000	1	SLIP COVER CONTOUR BACK 7" SM
36	A5580	A	1	ASMBLY LATERAL W/ ELBOW PAD MPS XS	73	A3048	000	1	SLIP COVER CONTOUR BACK 7" XS
37	T50086	000	1	ASMBLY CHEST VEST XS	74	A5630	000	1	ASMBLY ELBOW/ ARM PAD MPS XS
					75	A5678	000	1	ASMBLY CALF PAD 4.5" X 4.5"

Parts Detail-MPS (multi-position stander)

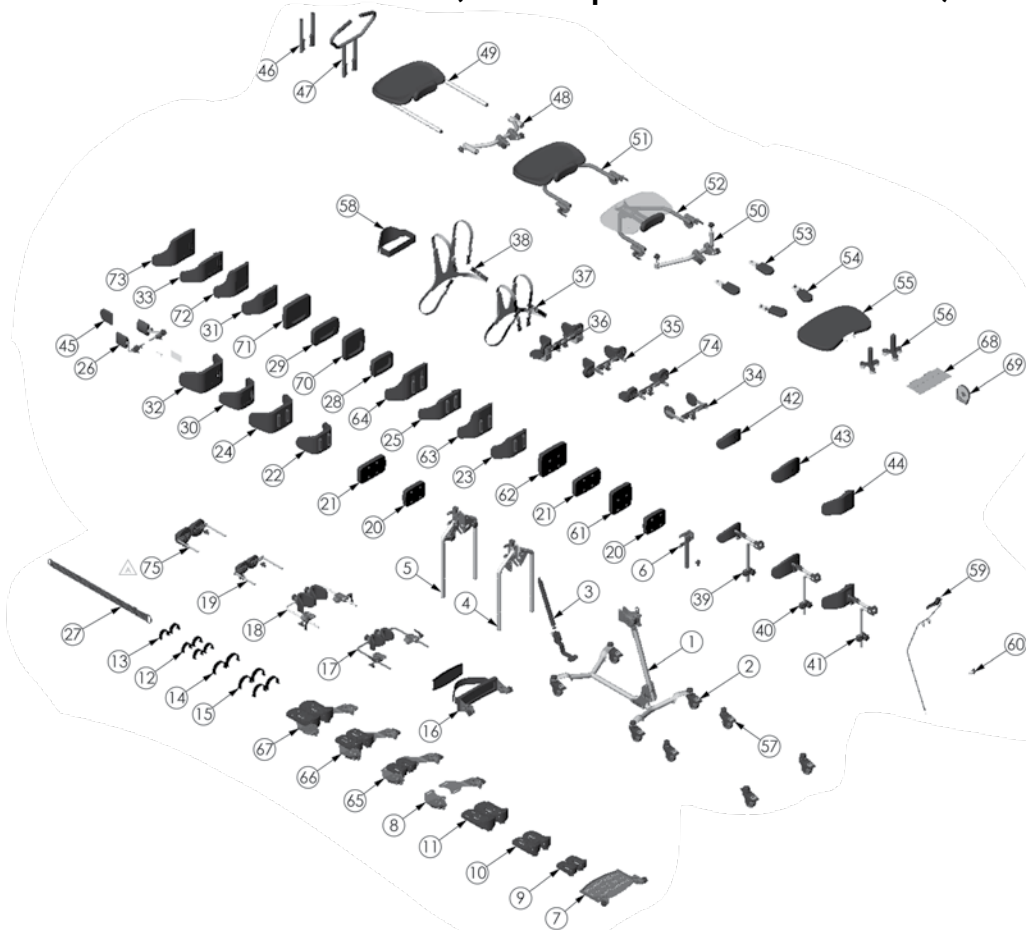


ITEM NO.	PART NO.	REV	QTY.	NAME	ITEM NO.	PART NO.	REV	QTY.	NAME
1	A5520	A	1	ASMBLY BASE ZING FORWARD PRONE	38	T50088	000	1	ASMBLY CHEST VEST SM
2	A5526	000	1	ASMBLY CASTERS BASIC MPS	39	Y5626	A	1	ASMBLY HEAD SUPPORT XS
3	A5528	000	1	ASMBLY GAS SPRING LIFT MPS	40	Y5628	A	1	ASMBLY HEAD SUPPORT SMALL
4	A5530	B	1	ASMBLY LEGS MPS STANDARD	41	A5586	000	1	ASMBLY HEAD REST CONTOUR SM
5	A5532	B	1	ASMBLY LEGS MPS ABDUCTION	42	A3035	000	1	SLIP COVER HEAD REST XS
6	A5534	000	1	ASMBLY MAST MPS	43	A3036	000	1	SLIP COVER HEAD REST SM
7	A5536	000	1	ASMBLY PLATFORM FOOT PLATE MPS	44	A3037	000	1	COVER HEAD REST CONTOUR SM
8	A5538	000	1	ASMBLY FOOT PLATE MPS MULTI-ADJ	45	A3039	000	1	SLIP COVER 3.5" LATERAL PAD
9	A5540	000	1	ASMBLY FOOT PLATE MPS EI	46	A5592	000	1	ASMBLY TRAY ASSEMBLY MOUNT MPS
10	A5542	000	1	ASMBLY FOOT PLATES XS	47	A5594	000	1	ASMBLY PUSH HANDLE MPS
11	A5544	000	1	ASMBLY FOOT PLATES SM	48	A5595	000	1	ASMBLY TABLE MOUNT BASIC MPS
12	A5546	000	1	ASMBLY SECURE FOOT STRAPS XS	49	A5596	000	1	ASMBLY TRAY MOLDED MPS
13	A5548	000	1	ASMBLY FOOT STRAPS XS	50	A5599	A	1	ASMBLY TABLE MOUNT TILT MPS
14	T50080	A	1	ASMBLY FOOT STRAP PED	51	A5600	A	1	ASMBLY TRAY SWING AWAY MPS
15	T50082	A	1	ASMBLY FOOT STRAP SECURE PED	52	A5602	A	1	ASMBLY TRAY SWING AWAY CLEAR MPS
16	A5550	000	1	ASMBLY KNEE PAD SOFT MPS	53	A5604	000	1	ASMBLY FOREARM WINGS MPS
17	A5556	000	1	ASMBLY KNEE MULTI ADJUSTABLE XS	54	A5606	000	1	ASMBLY FOREARM WINGS CLEAR MPS
18	A5558	000	1	ASMBLY KNEE MULTI-ADJUSTABLE SM	55	A3038	000	1	PADDED SLIP COVER TRAY MPS
19	A5560	000	1	ASMBLY CALF PAD INDEPENDENT MPS	56	NG50042	000	1	SUPPORT GRIP OPTION
20	A5562	000	1	ASMBLY PAD PLANAR 5" X 7"	57	A5608	000	1	ASMBLY CASTERS DIRECTIONAL MPS
21	A5564	000	1	ASMBLY PAD PLANAR 5" X 9"	58	A5610	000	1	ASMBLY CHEST STRAP MPS
22	A5566	000	1	ASMBLY CONTOUR PELVIC 5" XS	59	A5612	000	1	ASMBLY DUAL LIFT CONTROL MPS XS
23	A5568	000	1	ASMBLY CONTOUR BACK 5" XS	60	A5614	000	1	ASMBLY GAS SPRING LOCK OUT
24	A5570	000	1	ASMBLY CONTOUR PELVIC 5" SM	61	A5616	000	1	ASMBLY PAD PLANAR 7" X 7"
25	A5572	000	1	ASMBLY CONTOUR BACK 5" SM	62	A5618	000	1	ASMBLY PAD PLANAR 7" X 9"
26	A5574	000	1	ASMBLY HIP GUIDES MPS XS	63	A5620	000	1	ASMBLY CONTOUR BACK 7" XS
27	A3011	000	1	POSITIONING BELT ADJUSTABLE XS	64	A5622	000	1	ASMBLY CONTOUR BACK 7" SM
28	A3028	000	1	SLIP COVER PLANAR 5" X 7"	65	A5624	000	1	MULTI-ADJUSTABLE FOOT PLATE MPS EI
29	A3029	000	1	SLIP COVER PLANAR 5" X 9"	66	A5626	000	1	MULTI-ADJUSTABLE FOOT PLATE MPS SM
30	A3030	000	1	SLIP COVER CONTOUR PELVIC 5" XS	67	A5628	000	1	MULTI-ADJUSTABLE FOOT PLATE MPS SM
31	A3031	000	1	SLIP COVER CONTOUR BACK 5" XS	68	80809	A	1	TOOL POUCH
32	A3032	000	1	SLIP COVER CONTOUR PELVIC 5" SM	69	80246	000	1	SEAT ANGLE LOCATOR
33	A3033	000	1	SLIP COVER CONTOUR BACK 5" SM	70	A3045	000	1	SLIP COVER PLANAR 7" X 7"
34	A5576	A	1	ASMBLY LATERAL GUIDES MPS SM	71	A3046	000	1	SLIP COVER PLANAR 7" X 9"
35	A5578	A	1	ASMBLY LATERAL W/ ELBOW PAD MPS XS	72	A3047	000	1	SLIP COVER CONTOUR BACK 7" XS
36	A5580	A	1	ASMBLY LATERAL W/ ELBOW PAD MPS XS	73	A3048	000	1	SLIP COVER CONTOUR BACK 7" SM
37	T50086	000	1	ASMBLY CHEST VEST XS	74	A5630	000	1	ASMBLY ELBOW/ ARM PAD MPS XS
					75	A5678	000	1	ASMBLY CALF PAD 4.5" X 4.5"

Parts Detail-Tilt Table Supine

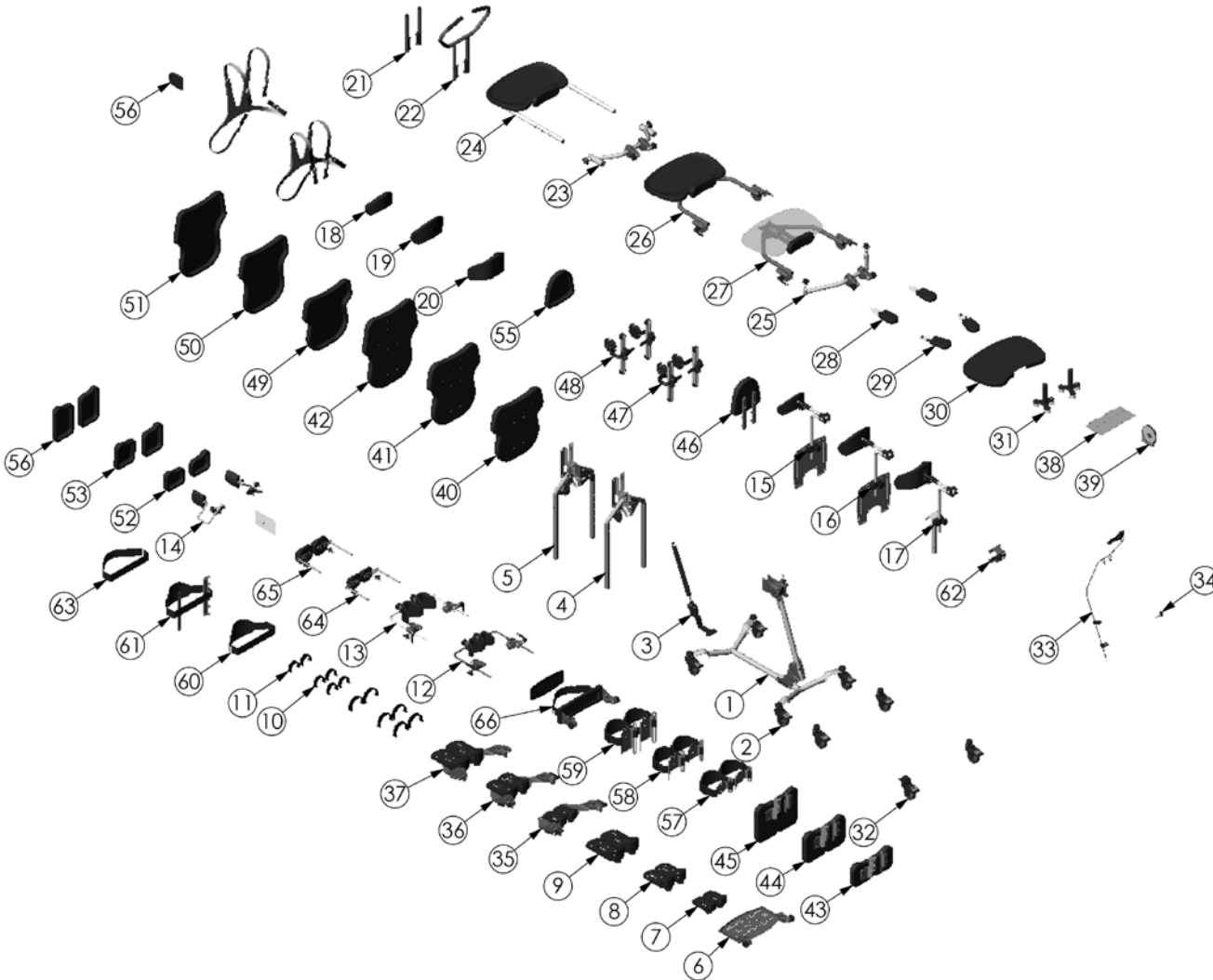


ITEM NO.	PART NO.	REV	QTY.	NAME
1	A5520	B	1	ASMBLY BASE ZING FORWARD PRONE
2	A5526	000	1	ASMBLY CASTERS BASIC MPS
3	A5528	A	1	ASMBLY GAS SPRING LIFT MPS
4	A5632	000	1	ASMBLY LEGS TT STANDARD
5	A5634	000	1	ASMBLY LEGS TT ABDUCTION
6	A5536	000	1	ASMBLY PLATFORM FOOT PLATE MPS
7	A5540	000	1	ASMBLY FOOT PLATE MPS EI
8	A5542	000	1	ASMBLY FOOT PLATES XS
9	A5544	000	1	ASMBLY FOOT PLATES SM
10	A5546	000	1	ASMBLY SECURE FOOT STRAPS XS
11	A5548	000	1	ASMBLY FOOT STRAPS XS
12	A5556	A	1	ASMBLY KNEE MULTI ADJUSTABLE XS
13	A5558	A	1	ASMBLY KNEE MULTI-ADJUSTABLE SM
14	A5636	000	1	ASMBLY HIP GUIDES TT XS
15	Y5626	B	1	ASMBLY HEAD SUPPORT XS
16	Y5628	B	1	ASMBLY HEAD SUPPORT SMALL
17	A5586	A	1	ASMBLY HEAD REST CONTOUR SM
18	A3035	000	1	SLIP COVER HEAD REST XS
19	A3036	000	1	SLIP COVER HEAD REST SM
20	A3037	000	1	COVER HEAD REST CONTOUR SM
21	A5592	000	1	ASMBLY TRAY ASSEMBLY MOUNT MPS
22	A5594	000	1	ASMBLY PUSH HANDLE MPS
23	A5595	A	1	ASMBLY TABLE MOUNT BASIC MPS
24	A5596	000	1	ASMBLY TRAY MOLDED MPS
25	A5599	A	1	ASMBLY TABLE MOUNT TILT MPS
26	A5600	B	1	ASMBLY TRAY SWING AWAY MPS
27	A5602	B	1	ASMBLY TRAY SWING AWAY CLEAR MPS
28	A5604	000	1	ASMBLY FOREARM WINGS MPS
29	A5606	000	1	ASMBLY FOREARM WINGS CLEAR MPS
30	A3038	000	1	PADDED SLIP COVER TRAY MPS
31	NG50042	000	1	SUPPORT GRIP OPTION
32	A5608	000	1	ASMBLY CASTERS DIRECTIONAL MPS
33	A5612	A	1	ASMBLY DUAL LIFT CONTROL MPS XS
34	A5614	000	1	ASMBLY GAS SPRING LOCK OUT
35	A5624	000	1	MULTI-ADJUSTABLE FOOT PLATE MPS EI
36	A5626	000	1	MULTI-ADJUSTABLE FOOT PLATE MPS XS
37	A5628	000	1	MULTI-ADJUSTABLE FOOT PLATE MPS SM
38	80809	A	1	TOOL POUCH
39	80246	000	1	SEAT ANGLE LOCATOR
40	A5638	000	1	ASMBLY PAD ZING PLANAR 14" X 14"
41	A5640	000	1	ASMBLY PAD ZING PLANAR 16" X 14"
42	A5642	000	1	ASMBLY PAD ZING PLANAR 18" X 14"
43	A5644	000	1	ASMBLY LEG PAD ZING TT 4"
44	A5646	000	1	ASMBLY LEG PAD ZING TT 6"
45	A5648	000	1	ASMBLY LEG PAD ZING TT 8"
46	A5650	A	1	ASMBLY HEAD PAD ZING PLANAR
47	A5652	000	1	ASMBLY LATERAL GUIDES TT 4" - 8"
48	A5654	000	1	ASMBLY LATERAL GUIDES TT 7" - 11"
49	A3063	000	1	SLIP COVER 14" X 14"
50	A3064	000	1	SLIP COVER 16" X 14"
51	A3065	000	1	SLIP COVER 18" X 14"
52	A5660	000	1	ASMBLY SLIP COVERS LEG PAD 4" TT
53	A5662	000	1	ASMBLY SLIP COVERS LEG PAD 6"
56	A5664	000	1	ASMBLY SLIP COVERS LEG PAD 8"
55	A3066	000	1	SLIP COVER PLANAR HEAD SUPPORT XS
56	A3039	000	1	SLIP COVER 3.5" LATERAL PAD
57	A5666	A	1	ASMBLY KNEE STRAP TT 4" PADS
58	A5668	A	1	ASMBLY KNEE STRAP TT 6" PADS
59	A5670	A	1	ASMBLY KNEE STRAP TT 8" PADS
60	A5672	000	1	ASMBLY STRAP W/PAD 25" - 38"
61	A5674	000	1	ASMBLY CHEST STRAP W/PAD TT
62	A5676	A	1	ASMBLY HEAD REST MOUNT ZING TT
63	A3062	000	1	POSITIONING BELT ADJUSTABLE TT
64	A5560	000	1	ASMBLY CALF PAD INDEPENDENT MPS
65	A5678	000	1	ASMBLY CALF PAD 4.5" X 4.5"
66	A5550	000	1	ASMBLY KNEE PAD SOFT MPS
67	T50080	A	1	ASMBLY FOOT STRAP PED
68	T50082	A	1	ASMBLY FOOT STRAP SECURE PED
69	T50086	000	1	ASMBLY CHEST VEST XS
70	T50088	000	1	ASMBLY CHEST VEST SM

Parts Detail-Tilt Table MPS (multi-position stander)

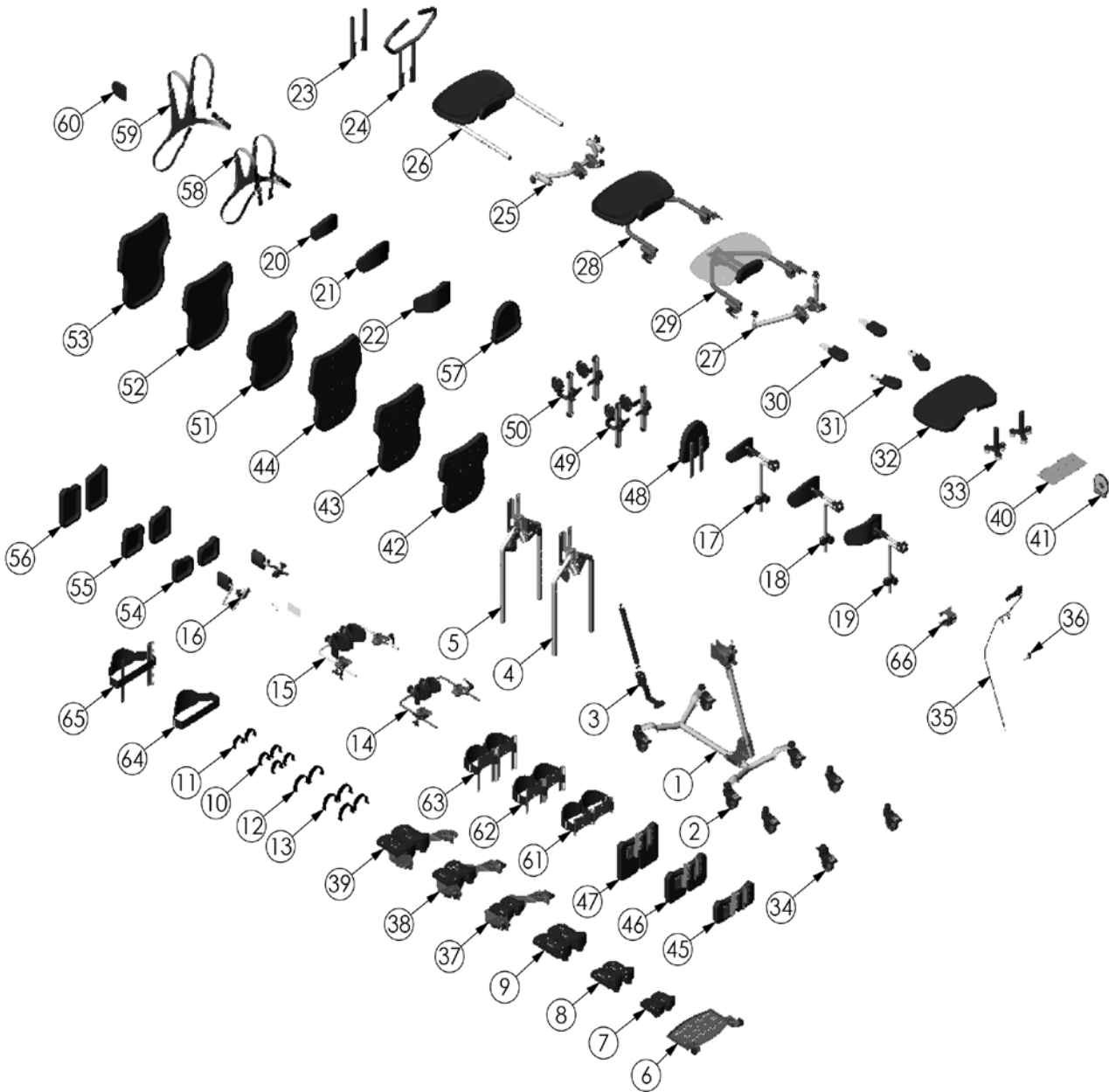

ITEM NO.	PART NO.	REV	QTY.	NAME
1	A5520	A	1	ASMBLY BASE ZING FORWARD PRONE
2	A5526	000	1	ASMBLY CASTERS BASIC MPS
3	A5528	000	1	ASMBLY GAS SPRING LIFT MPS
4	A5632	000	1	ASMBLY LEGS TT STANDARD
5	A5634	000	1	ASMBLY LEGS TT ABDUCTION
6	A5536	000	1	ASMBLY PLATFORM FOOT PLATE MPS
7	A5540	000	1	ASMBLY FOOT PLATE MPS EI
8	A5542	000	1	ASMBLY FOOT PLATES XS
9	A5544	000	1	ASMBLY FOOT PLATES SM
10	A5546	000	1	ASMBLY SECURE FOOT STRAPS XS
11	A5548	000	1	ASMBLY FOOT STRAPS XS
12	T50080	A	1	ASMBLY FOOT STRAP PED
13	T50082	A	1	ASMBLY FOOT STRAP SECURE PED
14	A5556	000	1	ASMBLY KNEE MULTI-ADJUSTABLE XS
15	A5558	000	1	ASMBLY KNEE MULTI-ADJUSTABLE SM
16	A5636	000	1	ASMBLY HIP GUIDES TT XS
17	Y5626	A	1	ASMBLY HEAD SUPPORT XS
18	Y5628	A	1	ASMBLY HEAD SUPPORT SMALL
19	A5586	000	1	ASMBLY HEAD REST CONTOUR SM
20	A3035	000	1	SLIP COVER HEAD REST XS
21	A3036	000	1	SLIP COVER HEAD REST SM
22	A3037	000	1	COVER HEAD REST CONTOUR SM
23	A5592	000	1	ASMBLY TRAY ASSEMBLY MOUNT MPS
24	A5594	000	1	ASMBLY PUSH HANDLE MPS
25	A5595	000	1	ASMBLY TABLE MOUNT BASIC MPS
26	A5596	000	1	ASMBLY TRAY MOLDED MPS
27	A5599	A	1	ASMBLY TABLE MOUNT TILT MPS
28	A5600	A	1	ASMBLY TRAY SWING AWAY MPS
29	A5602	A	1	ASMBLY TRAY SWING AWAY CLEAR MPS
30	A5604	000	1	ASMBLY FOREARM WINGS MPS
31	A5606	000	1	ASMBLY FOREARM WINGS CLEAR MPS
32	A3038	000	1	PADDED SLIP COVER TRAY MPS
33	NG50042	000	1	SUPPORT GRIP OPTION
34	A5608	000	1	ASMBLY CASTERS DIRECTIONAL MPS
35	A5612	000	1	ASMBLY DUAL LIFT CONTROL MPS XS
36	A5614	000	1	ASMBLY GAS SPRING LOCK OUT
37	A5624	000	1	MULTI-ADJUSTABLE FOOT PLATE MPS EI
38	A5626	000	1	MULTI-ADJUSTABLE FOOT PLATE MPS XS
39	A5628	000	1	MULTI-ADJUSTABLE FOOT PLATE MPS SM
40	80809	A	1	TOOL POUCH
41	80246	000	1	SEAT ANGLE LOCATOR
42	A5638	000	1	ASMBLY PAD ZING PLANAR 14" X 14"
43	A5640	000	1	ASMBLY PAD ZING PLANAR 16" X 14"
44	A5642	000	1	ASMBLY PAD ZING PLANAR 18" X 14"
45	A5644	000	1	ASMBLY LEG PAD ZING TT 4"
46	A5646	000	1	ASMBLY LEG PAD ZING TT 6"
47	A5648	000	1	ASMBLY LEG PAD ZING TT 8"
48	A5650	000	1	ASMBLY HEAD PAD ZING PLANAR
49	A5652	000	1	ASMBLY LATERAL GUIDES TT 4" - 8"
50	A5654	000	1	ASMBLY LATERAL GUIDES TT 7" - 11"
51	A3063	000	1	SLIP COVER 14" X 14"
52	A3064	000	1	SLIP COVER 16" X 14"
53	A3065	000	1	SLIP COVER 18" X 14"
54	A5660	000	1	ASMBLY SLIP COVERS LEG PAD 4" TT
55	A5662	000	1	ASMBLY SLIP COVERS LEG PAD 6"
56	A5664	000	1	ASMBLY SLIP COVERS LEG PAD 8"
57	A3066	000	1	SLIP COVER PLANAR HEAD SUPPORT XS
58	T50086	000	1	ASMBLY CHEST VEST XS
59	T50088	000	1	ASMBLY CHEST VEST SM
60	A3039	000	1	SLIP COVER 3.5" LATERAL PAD
61	A5666	000	1	ASMBLY KNEE STRAP TT 4" PADS
62	A5668	000	1	ASMBLY KNEE STRAP TT 6" PADS
63	A5670	000	1	ASMBLY KNEE STRAP TT 8" PADS
64	A5672	000	1	ASMBLY STRAP W/PAD 25" - 38"
65	A5674	000	1	ASMBLY CHEST STRAP W/PAD TT
66	A5676	000	1	ASMBLY HEAD REST MOUNT ZING TT

Parts Detail-Prone

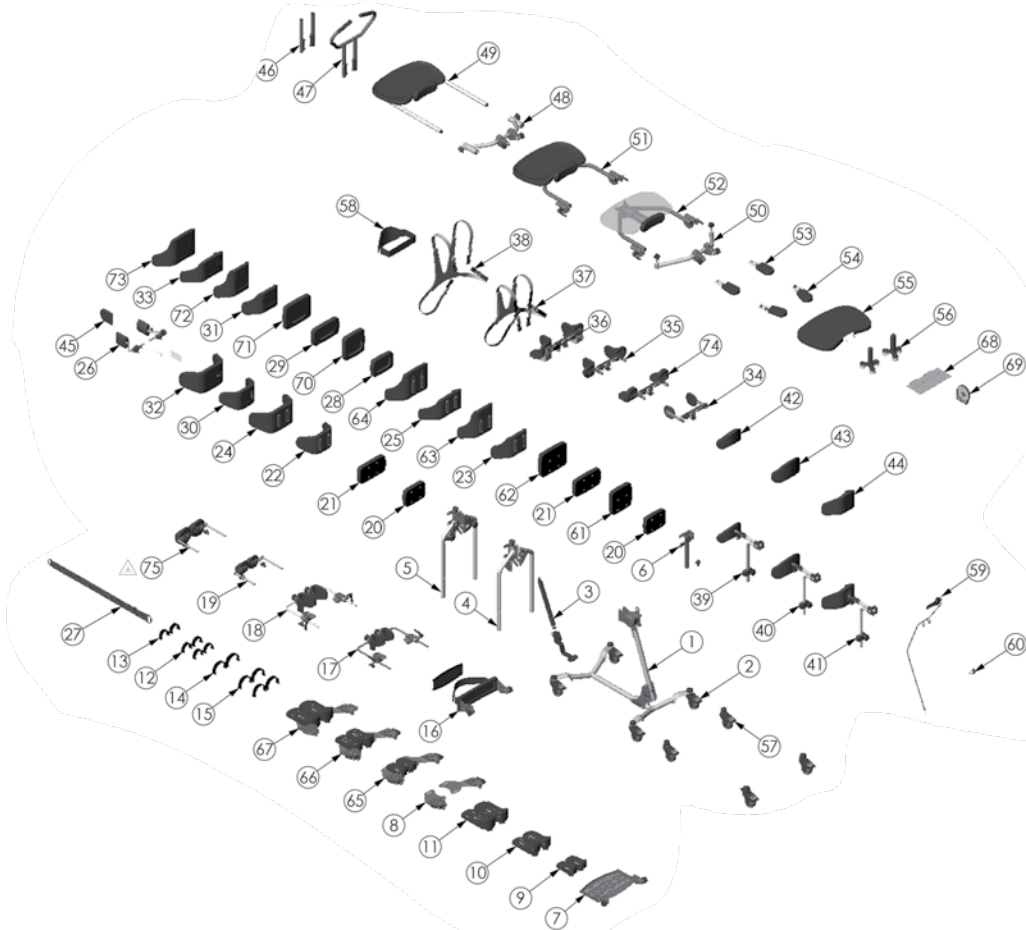

ITEM NO.	PART NO.	REV	QTY.	NAME	ITEM NO.	PART NO.	REV	QTY.	NAME
1	A5520	A	1	ASMBLY BASE ZING FORWARD PRONE	38	T50088	000	1	ASMBLY CHEST VEST SM
2	A5528	000	1	ASMBLY CASTERS BASIC MPS	39	Y5626	A	1	ASMBLY HEAD SUPPORT XS
3	A5528	000	1	ASMBLY GAS SPRING LIFT MPS	40	Y5628	A	1	ASMBLY HEAD SUPPORT SMALL
4	A5530	B	1	ASMBLY LEGS MPS STANDARD	41	A5586	000	1	ASMBLY HEAD REST CONTOUR SM
5	A5532	B	1	ASMBLY LEGS MPS ABDUCTION	42	A3035	000	1	SLIP COVER HEAD REST XS
6	A5534	000	1	ASMBLY MAST MPS	43	A3036	000	1	SLIP COVER HEAD REST SM
7	A5536	000	1	ASMBLY PLATFORM FOOT PLATE MPS	44	A3037	000	1	COVER HEAD REST CONTOUR SM
8	A5538	000	1	ASMBLY FOOT PLATE MPS MULTI-ADJ	45	A3039	000	1	SLIP COVER 3.5" LATERAL PAD
9	A5540	000	1	ASMBLY FOOT PLATE MPS EI	46	A5592	000	1	ASMBLY TRAY ASSEMBLY MOUNT MPS
10	A5542	000	1	ASMBLY FOOT PLATES XS	47	A5594	000	1	ASMBLY PUSH HANDLE MPS
11	A5544	000	1	ASMBLY FOOT PLATES SM	48	A5595	000	1	ASMBLY TABLE MOUNT BASIC MPS
12	A5546	000	1	ASMBLY SECURE FOOT STRAPS XS	49	A5596	000	1	ASMBLY TRAY MOLDED MPS
13	A5548	000	1	ASMBLY FOOT STRAPS XS	50	A5599	A	1	ASMBLY TABLE MOUNT TILT MPS
14	T50080	A	1	ASMBLY FOOT STRAP PED	51	A5600	A	1	ASMBLY TRAY SWING AWAY MPS
15	T50082	A	1	ASMBLY FOOT STRAP SECURE PED	52	A5602	A	1	ASMBLY TRAY SWING AWAY CLEAR MPS
16	A5550	000	1	ASMBLY KNEE PAD SOFT MPS	53	A5604	000	1	ASMBLY FOREARM WINGS MPS
17	A5556	000	1	ASMBLY KNEE MULTI ADJUSTABLE XS	54	A5606	000	1	ASMBLY FOREARM WINGS CLEAR MPS
18	A5558	000	1	ASMBLY KNEE MULTI-ADJUSTABLE SM	55	A3038	000	1	PADDED SLIP COVER TRAY MPS
19	A5560	000	1	ASMBLY CALF PAD INDEPENDENT MPS	56	NG50042	000	1	SUPPORT GRIP OPTION
20	A5562	000	1	ASMBLY PAD PLANAR 5" X 7"	57	A5608	000	1	ASMBLY CASTERS DIRECTIONAL MPS
21	A5564	000	1	ASMBLY PAD PLANAR 5" X 9"	58	A5610	000	1	ASMBLY CHEST STRAP MPS
22	A5566	000	1	ASMBLY CONTOUR PELVIC 5" XS	59	A5612	000	1	ASMBLY DUAL LIFT CONTROL MPS XS
23	A5568	000	1	ASMBLY CONTOUR BACK 5" XS	60	A5614	000	1	ASMBLY GAS SPRING LOCK OUT
24	A5570	000	1	ASMBLY CONTOUR PELVIC 5" SM	61	A5616	000	1	ASMBLY PAD PLANAR 7" X 7"
25	A5572	000	1	ASMBLY CONTOUR BACK 5" SM	62	A5618	000	1	ASMBLY PAD PLANAR 7" X 9"
26	A5574	000	1	ASMBLY HIP GUIDES MPS XS	63	A5620	000	1	ASMBLY CONTOUR BACK 7" XS
27	A3011	000	1	POSITIONING BELT ADJUSTABLE XS	64	A5622	000	1	ASMBLY CONTOUR BACK 7" SM
28	A3028	000	1	SLIP COVER PLANAR 5" X 7"	65	A5624	000	1	MULTI-ADJUSTABLE FOOT PLATE MPS EI
29	A3029	000	1	SLIP COVER PLANAR 5" X 9"	66	A5626	000	1	MULTI-ADJUSTABLE FOOT PLATE MPS SM
30	A3030	000	1	SLIP COVER CONTOUR PELVIC 5" XS	67	A5628	000	1	MULTI-ADJUSTABLE FOOT PLATE MPS SM
31	A3031	000	1	SLIP COVER CONTOUR BACK 5" XS	68	80809	A	1	TOOL POUCH
32	A3032	000	1	SLIP COVER CONTOUR PELVIC 5" SM	69	80246	000	1	SEAT ANGLE LOCATOR
33	A3033	000	1	SLIP COVER CONTOUR BACK 5" SM	70	A3045	000	1	SLIP COVER PLANAR 7" X 7"
34	A5576	A	1	ASMBLY LATERAL GUIDES MPS SM	71	A3046	000	1	SLIP COVER PLANAR 7" X 9"
35	A5578	A	1	ASMBLY LATERAL W/ ELBOW PAD MPS XS	72	A3047	000	1	SLIP COVER CONTOUR BACK 7" XS
36	A5580	A	1	ASMBLY LATERAL W/ ELBOW PAD MPS XS	73	A3048	000	1	SLIP COVER CONTOUR BACK 7" SM
37	T50086	000	1	ASMBLY CHEST VEST XS	74	A5630	000	1	ASMBLY ELBOW/ ARM PAD MPS XS
					75	A5678	000	1	ASMBLY CALF PAD 4.5" X 4.5"

Maintenance

Cleaning/Disinfecting

- Clean the EasyStand with a mild, non-abrasive disinfectant or soap and water.
- Check the EasyStand at least weekly to ensure all nuts, bolts and adjustable parts are tightened securely, failure to do so may result in parts falling off which may cause a choking hazard for children.
- Do not use bleach or any other bleach-based cleaners on the EasyStand.
- Disinfecting wipes are safe to use on metal and non-porous surfaces.
- Use a damp sponge with mild soap or a mild, non-abrasive disinfectant.
- Spot clean Neoprene with mild soap and water.
- Hygienic covers may be removed and hand washed with water. Air dry the covers. Always use care when removing and replacing the covers.
- Do not dry clean any upholstery items.
- If the EasyStand is being used with one user, clean the EasyStand on a monthly basis or as needed.
- If the EasyStand is being used in a multiple-user environment, the EasyStand should be cleaned after each use.

Urethane Coated Fabrics and Hygienic Washable Covers (Dartex or generic equivalent)

- The recommended cleaning and disinfection instructions should be followed whenever possible to avoid damage to the PU membrane which could lead to premature failure of the cover.
- For superficial dirt, use a disposable wipe and a warm solution of neutral detergent. Do not use abrasive cleaners.
- Disinfect using a 0.1% sodium hypochlorite solution (1000ppm chlorine), stronger bleach solutions not recommended.
- Thoroughly rinse any remaining cleaning solution with water and dry fabric before re-use or storage.
- Do not iron.

Limited Warranty

This warranty is extended only to the original purchaser/customer (or supplier non consumer who does not buy or resell). Altimate Medical, Inc., warrants the EasyStand against defects in materials and workmanship as listed below.

- Steel Frames - 5 year warranty
- Upholstered components, plastic pads, rubber parts, painted surfaces, bearings and other parts not specifically identified above - 90 day warranty
- Gas Spring - 2 year warranty

The warranty period commences on the original ship date from Altimate Medical. Any product proven to Altimate Medical satisfaction to be defective and within warranty period, shall be repaired or replaced at Altimate Medical's option.

ALTIMATE MEDICAL SOLE OBLIGATION AND YOUR EXCLUSIVE REMEDY UNDER THIS WARRANTY SHALL BE LIMITED TO SUCH REPAIR AND/OR REPLACEMENT. This warranty does not include any labor charges incurred in replacement parts installation. Freight charges to factory are at the expense of consumer to seller. Return freight charges will be prepaid by Altimate Medical. For warranty service, please contact the authorized supplier from whom you purchased your Altimate Medical product. In the event you do not receive satisfactory warranty service, please write directly to Altimate Medical at the address below. Do not return products to our factory without prior authorization.

LIMITATIONS AND EXCLUSIONS: The foregoing warranty shall not apply to serial numbered products if the serial number has been removed or defaced. Products subject to negligence, abuse, misuse, improper operation, improper maintenance, improper storage, or damages beyond Altimate Medical's control are not covered by this warranty, and that evaluation will be solely determined by Altimate Medical. Products modified without Altimate Medical's express written consent (including, but not limited to, modification through the use of unauthorized parts or attachments) are not covered by this warranty. The warranty shall not apply to problems arising from normal wear or failure to follow instructions. The warranty will be void if parts not manufactured by Altimate Medical or parts not complying with original equipment specifications are added to an Altimate Medical product.

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES. ALTIMATE MEDICAL SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

Altimate Medical maintains a policy of continual product improvement and reserves the right to change features, specifications, and prices without prior notification. Check with Altimate Medical for latest information.

Altimate Medical, Inc., 262 W. 1st St., Morton, MN 56270 USA
Phone: 507.697.6393 Toll Free: 800.342.8968 Fax: 507.697.6900
easystand.com info@easystand.com

Environmental Issues

After life of the product:

- Recycle in accordance with the recycling codes of your country.
- Recycle all metal parts.
- Plastic parts should be disposed of or recycled according to the directive in your country.
- Please inquire at your local disposal management and recycling center for local codes.
- Return gas spring cylinder to Altimate Medical, Inc. if no local recycling is available.